

Image Export to Pre-Defined Folder

New panel on tab |Third party Tools| in system options to define a standard export folder for image files using Mergefields.


Example Interface IEE-Systems (Dr. Visse)

- <https://www.iie-systems.de/>
- <https://www.dr-visse.de/>

The system creates web-based presentations from images saved in a linked folder.

The patient/partner receives a download link to view the images in a album or in a roundabout sequence.


To hand over patient image from Onyx (currently 2D only), on system options tab |Third Party Tools| ae export folder has to be defined using space holders:

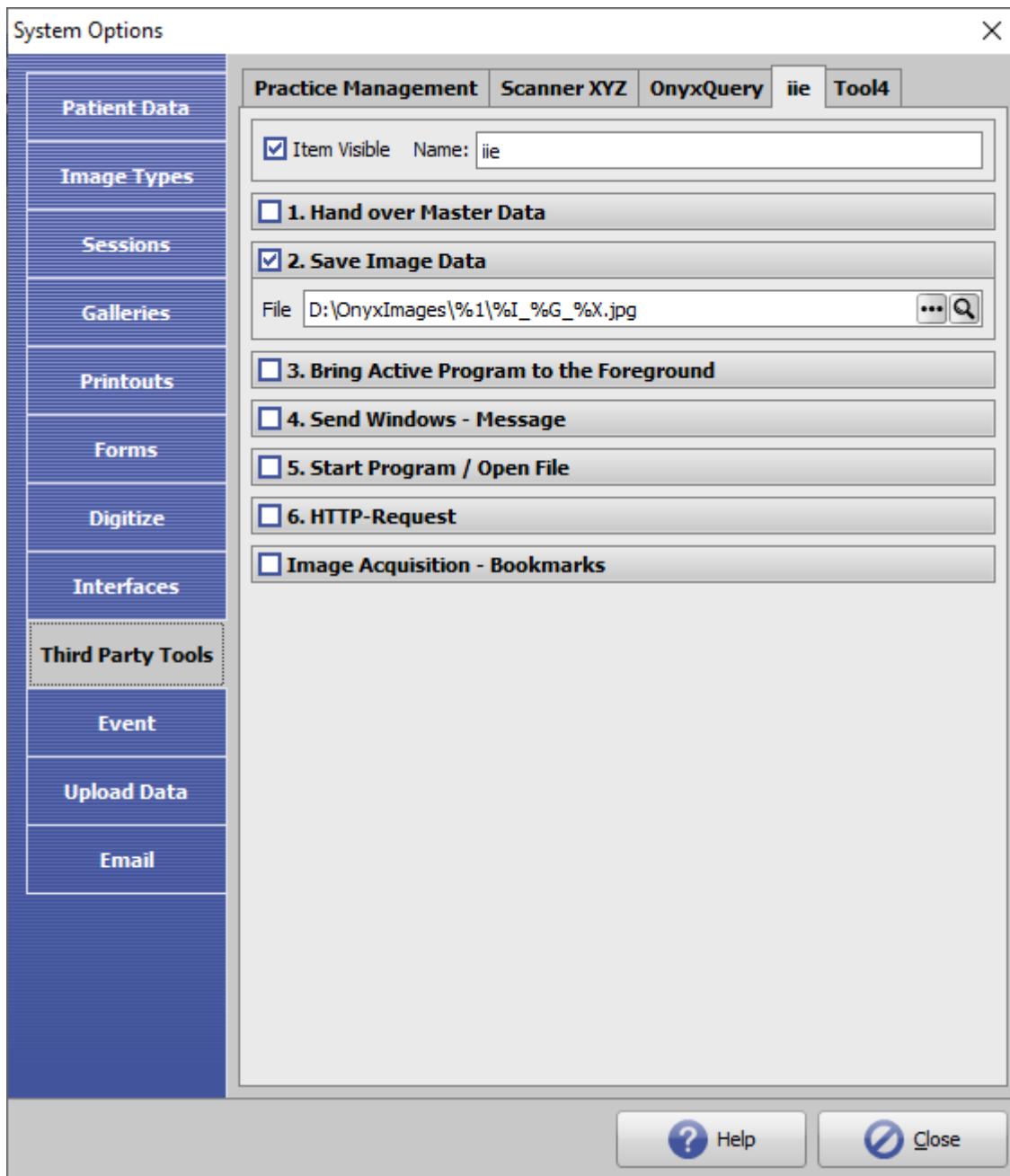

Useful Space holders ([Mergefields](#)):

Short cut	Meaning
%L	Index 1,2,3, ...
%6	PicFName
%7	PicCode (PatID+System code)
%G	Name of the session
%I	Image Type

To use the function, select images on tab |Images / 2D-Data| and strat transfer to folder by menu item [GoTo ...]

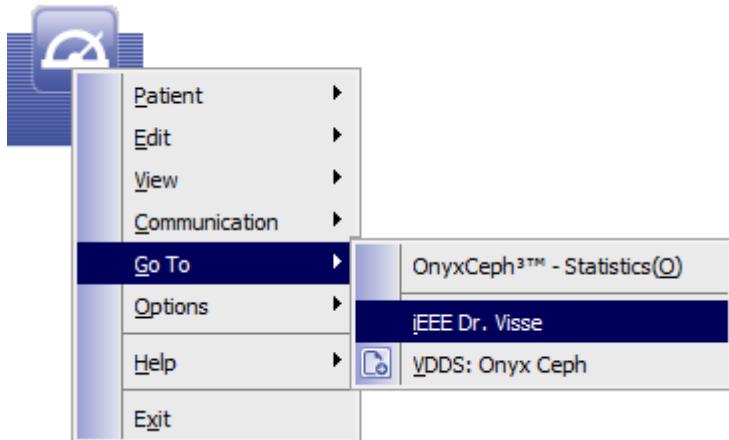

From:
<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:
<http://onyxwiki.net/doku.php?id=en:4657&rev=1624257633>

Last update: **2021/06/21 08:40**

