




Programmversion	BASIC	2D PRO	3D PRO	LAB	OMS
Verfügbarkeit	✗	Cloud icon	Cloud icon	Cloud icon	Cloud icon

In module Add Image 2D, 2D data (photos, xrays) can be imported from various sources. Record dates, [sessions](#) and [image types](#) can be assigned, [image attributes and comments](#) can be added and finally, the image can be saved in the Onyx database. Images can be imported individually or in blocks (batch import) and the appropriate patient can also be selected automatically using the QR code option.

First Steps

1. Activate "Clipping (Line)" in tab "Clip Object"
2. Place Landmarks at the model border, last point close to the first one
3. Press button "Trim" and confirm the correct trimming
4. Select and add socket in tab "Model Base"
5. Save as new finding

Module user interface

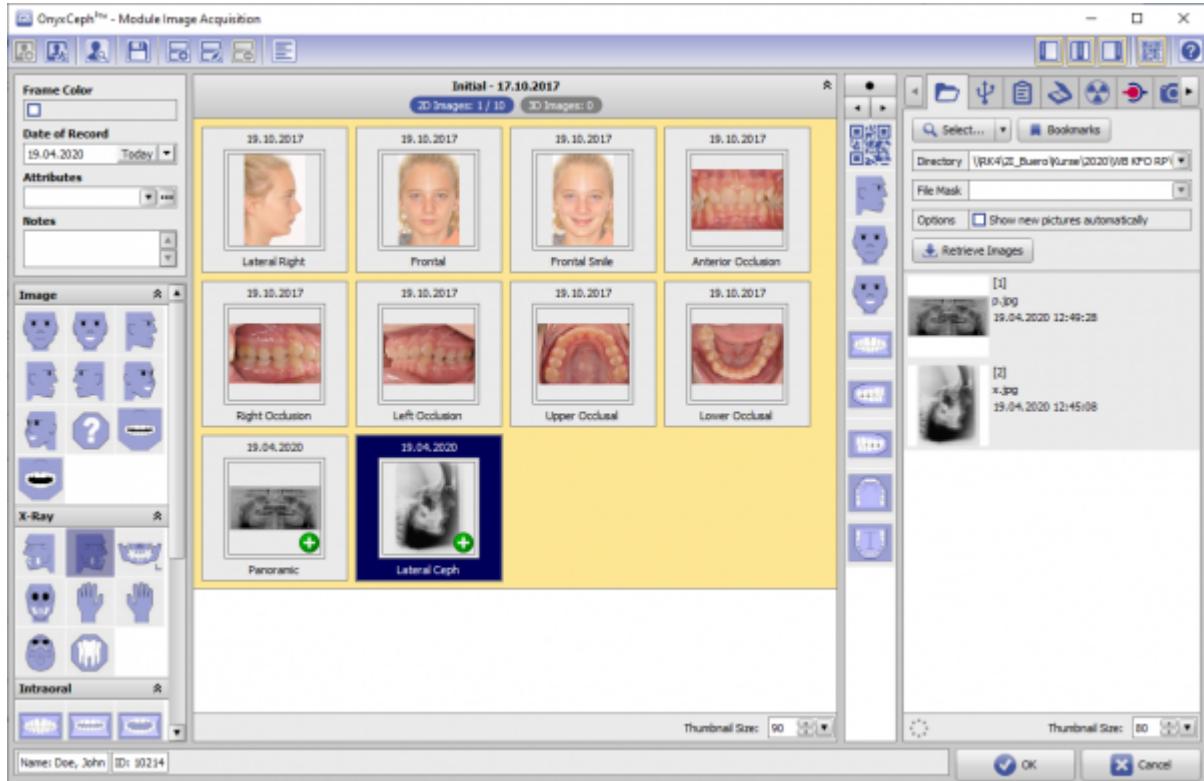

Links



https://image-instruments.de/Onyx3/Handout_Digitize_Images_2D_EN.pdf

From:
<http://onyxwiki.net/> - [OnyxCeph^{3™} Wiki]

Permanent link:
http://onyxwiki.net/doku.php?id=en:add_image_2d&rev=1586008977

Last update: **2020/04/04 16:02**

