


Module Aligner 3D: Revision 3.2.169

Attachment list

The attachment list in module [Aligner 3D](#) has been slightly revised visually and functionally on release 3.2.157+:

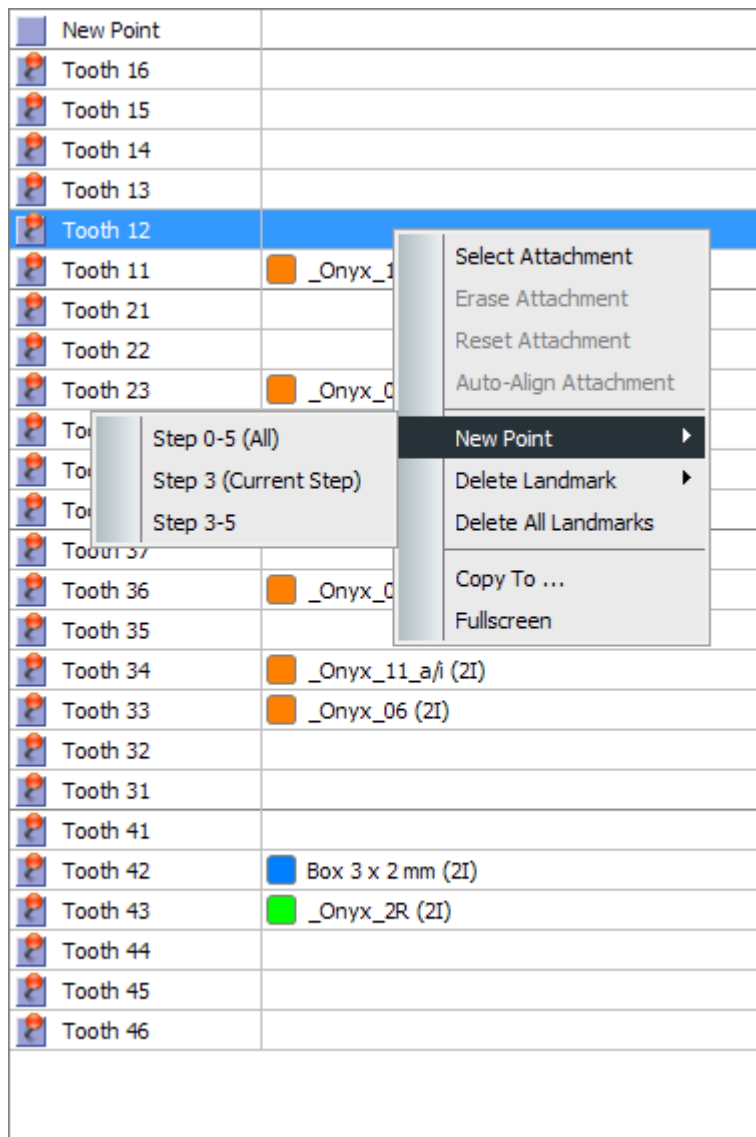


- New points (for additional attachments) can now be created for multiple steps - optionally for
 - all steps
 - all steps from step 0 to the current step
 - all steps from the current step to the Tx goal
- Deleting points can also be done selectively for
 - all steps
 - all steps from step 0 to the current step
 - all steps from the current step to the Tx goal
- new points are taken into account when copying
- menu items in the attachment list context menu have been prioritized
- at initialization of the attachment list not segmented crowns are omitted

- column manufacturer hidden
- attachment color is displayed
- visual separation between dental arch quadrants

Timeline

In addition to individualizing the name and comment of an aligner planning step, the step markers can now be assigned a color.

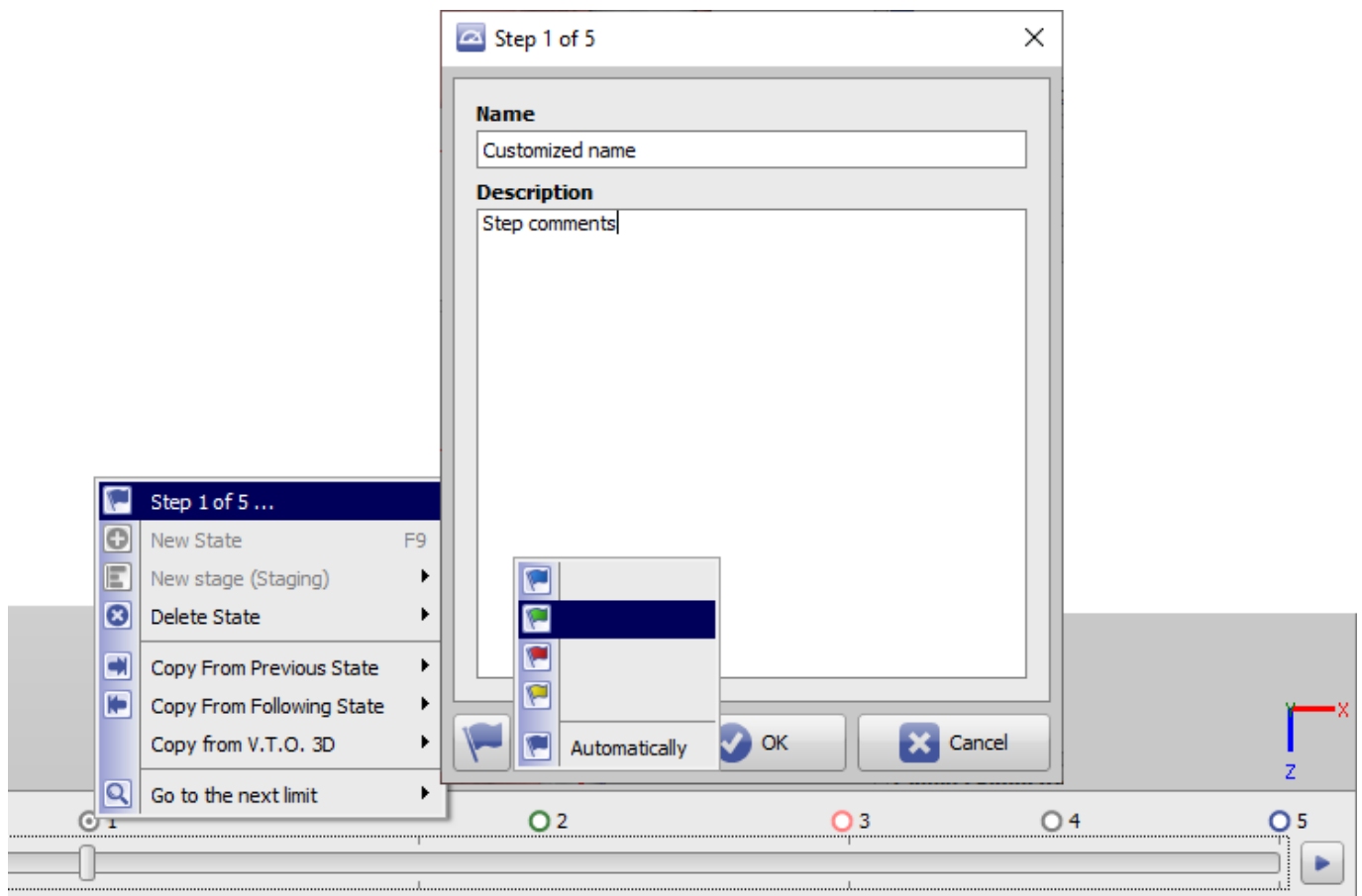

The feature can be used, for example, to visually distinguish stages and steps from each other.\

From:
<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:
<http://onyxwiki.net/doku.php?id=en:attmtlist>

Last update: **2021/11/30 08:57**

