

Module Aligner 3D: Revision 157+

Attachment list

The attachment list in module [Aligner 3D](#) has been slightly revised visually and functionally on release 3.2.157+:


New Point	
Tooth 16	
Tooth 15	
Tooth 14	
Tooth 13	
Tooth 12	
Tooth 11	_Onyx_1
Tooth 21	
Tooth 22	
Tooth 23	_Onyx_0
Tooth 37	
Tooth 36	_Onyx_0
Tooth 35	
Tooth 34	_Onyx_11_a/i (2I)
Tooth 33	_Onyx_06 (2I)
Tooth 32	
Tooth 31	
Tooth 41	
Tooth 42	Box 3 x 2 mm (2I)
Tooth 43	_Onyx_2R (2I)
Tooth 44	
Tooth 45	
Tooth 46	

- New points (for additional attachments) can now be created for multiple steps - optionally for
 - all steps
 - all steps from step 0 to the current step
 - all steps from the current step to the Tx goal
- Deleting points can also be done selectively for
 - all steps
 - all steps from step 0 to the current step
 - all steps from the current step to the Tx goal
- new points are taken into account when copying
- menu items in the attachment list context menu have been prioritized
- at initialization of the attachment list not segmented crowns are omitted

- column manufacturer hidden
- attachment color is displayed
- visual separation between dental arch quadrants

Timeline

In addition to individualizing the name and comment of an aligner planning step, the step markers can now be assigned a color.

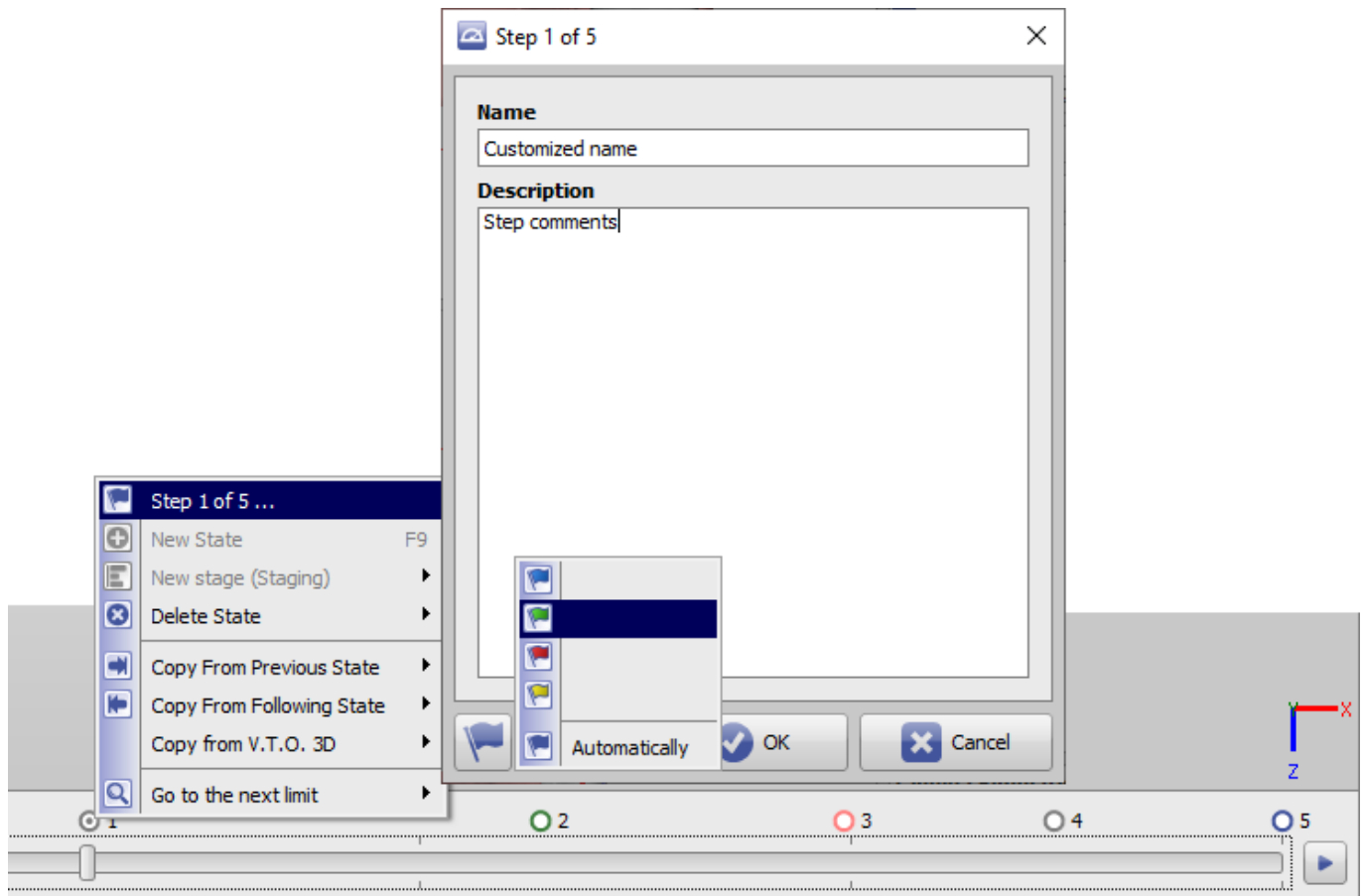

The feature can be used, for example, to visually distinguish stages and steps from each other.\

From:
<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:
<http://onyxwiki.net/doku.php?id=en:attmtlist&rev=1628597101>

Last update: **2021/08/10 14:05**

