

Module Bonding Jigs 3D



Program version	BASIC	2D PRO	3D PRO	LAB	OMS
Availablility	✗	✗			


- nicht
enthalten
- immer
enthalten
- optional
enthalten

In module Bonding Jigs 3D module, occlusal transfer jigs can be constructed to transfere the virtually planned bracket positions using metallic standard bracket adapters.

First Steps

1. Open Module with bracket-attached Model
2. Select Crowns
3. Assign adapter and Jig parameters to crowns
4. Construct Jigs individually or connected
5. Arrange Jigs for manufacturing / export
6. Save planning as project and/or as finding

Module user interface

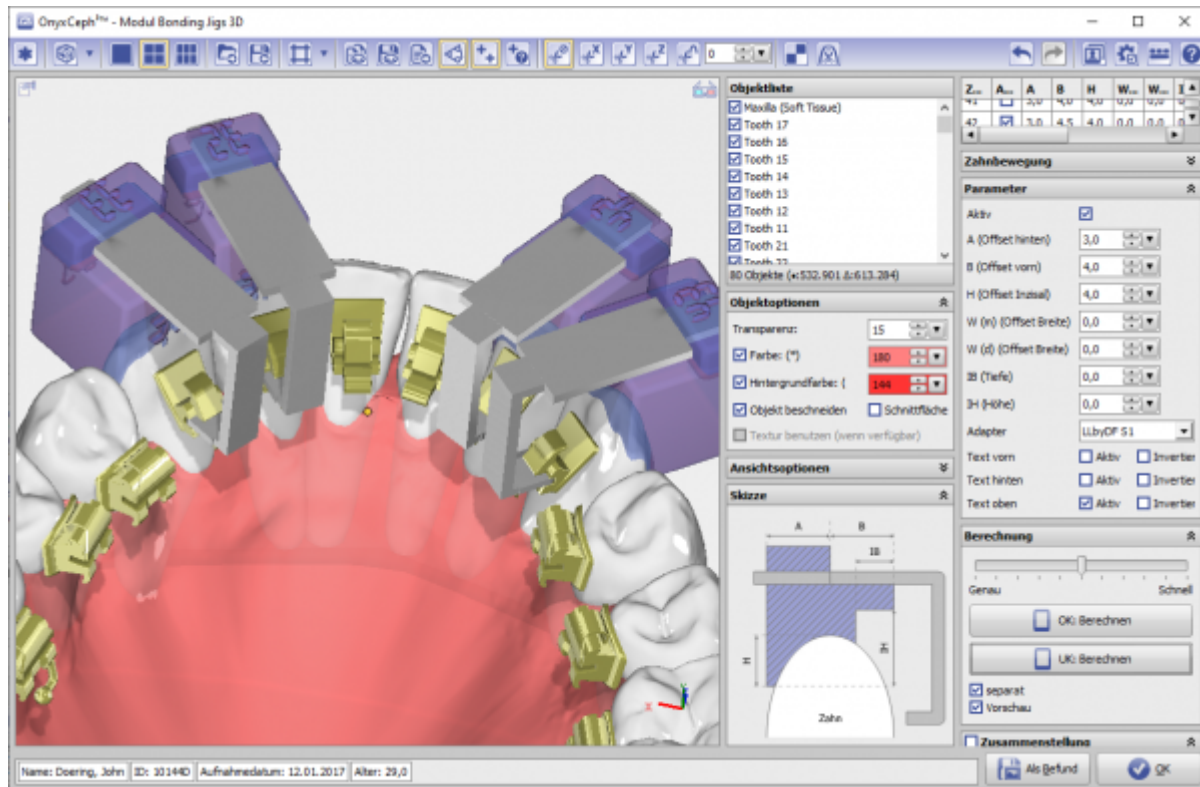

Links



From:
<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:
http://onyxwiki.net/doku.php?id=en:bonding_jigs_3d&rev=1644236695

Last update: **2022/02/07 13:24**

