

Module Combine | Register 3D



Program version	BASIC	2D PRO	3D PRO	LAB	OMS
Availability	✗	✗			

✗

not included

included by

default

optionally


included

The Combine 3D module provides 2 separate modes for the combination of two 3D findings based on overlapping features to allow additional options for visualization, diagnostics or planning.

First Steps

1. Open module with 2 findings
2. Select corresponding reference points
3. For ICP registration, select surface region
4. Switch to tab |Combine| and configure result
5. Save result as finding

Module user interface

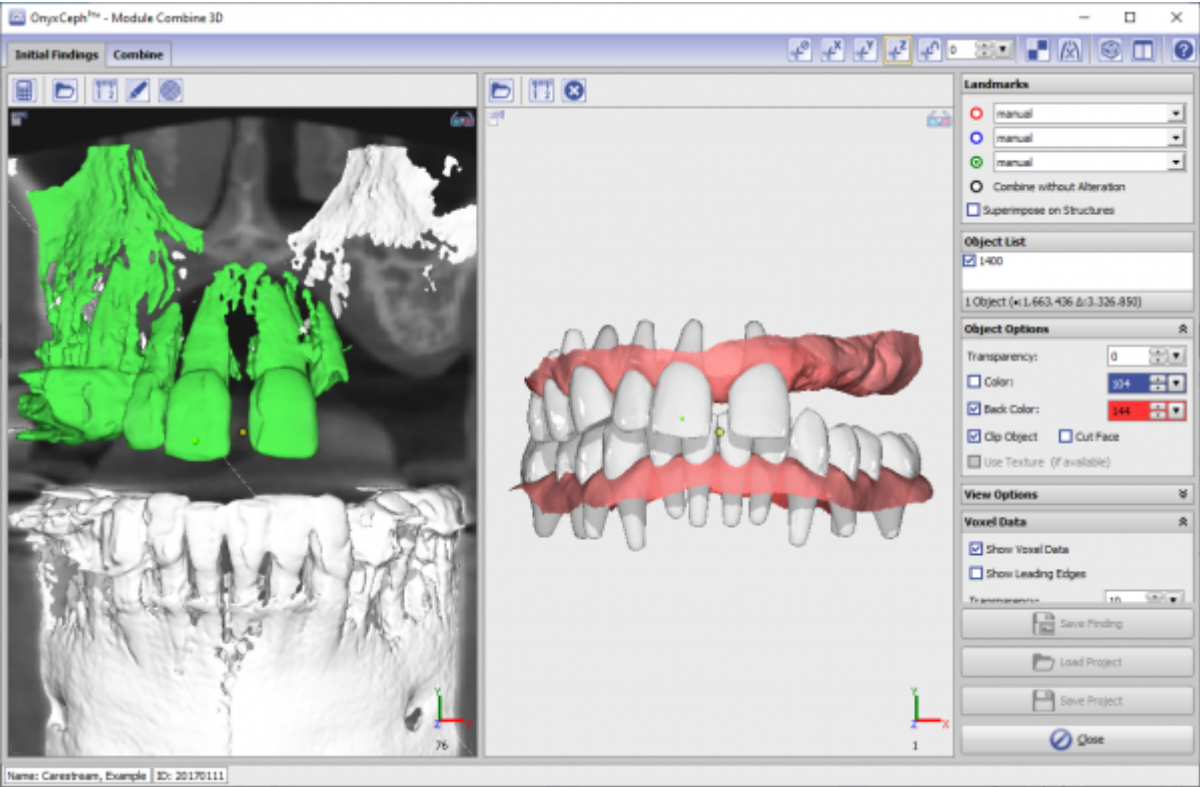

Verweise

	Combine 3D
	Modul-Handout Register 3D
	Panel Surface Registration 3D

From:

<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:

http://onyxwiki.net/doku.php?id=en:combine_3d

Last update:

2024/03/14 14:51

