

Module Combine 3D



Program version	BASIC	2D PRO	3D PRO	LAB	OMS
Availability					

 not included

 included by

default

 optionally


included

The Combine 3D module provides 2 separate modes for the combination of two 3D findings based on overlapping features to allow additional options for visualization, diagnostics or planning.

First Steps

1. Open module with 2 findings
2. Select corresponding reference points
3. For ICP registration, select surface region
4. Switch to tab |Combine| and configure result
5. Save result as finding

Module user interface

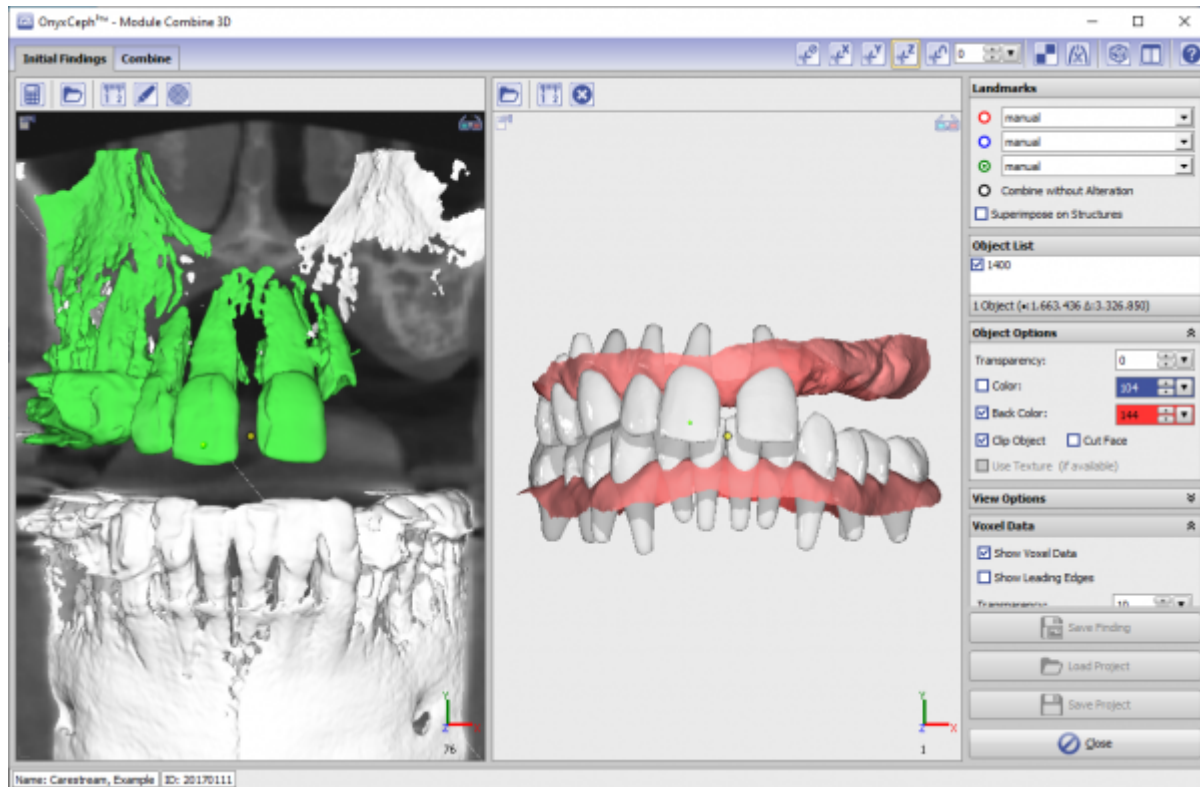

Verweise



From:
<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:
http://onyxwiki.net/doku.php?id=en:combine_3d&rev=1660810928

Last update: **2022/08/18 10:22**

