


Individualize Bracket Base

If the individual crown curvature deviates from the tooth shape used for programming the bracket base, the bracket base can be individualized by varying the angle between the straight-wire arc plane and FACC while **keeping the incisal edge distance unchanged** (5.04 mm in the example) compared to the initial "best-fit position" (green).

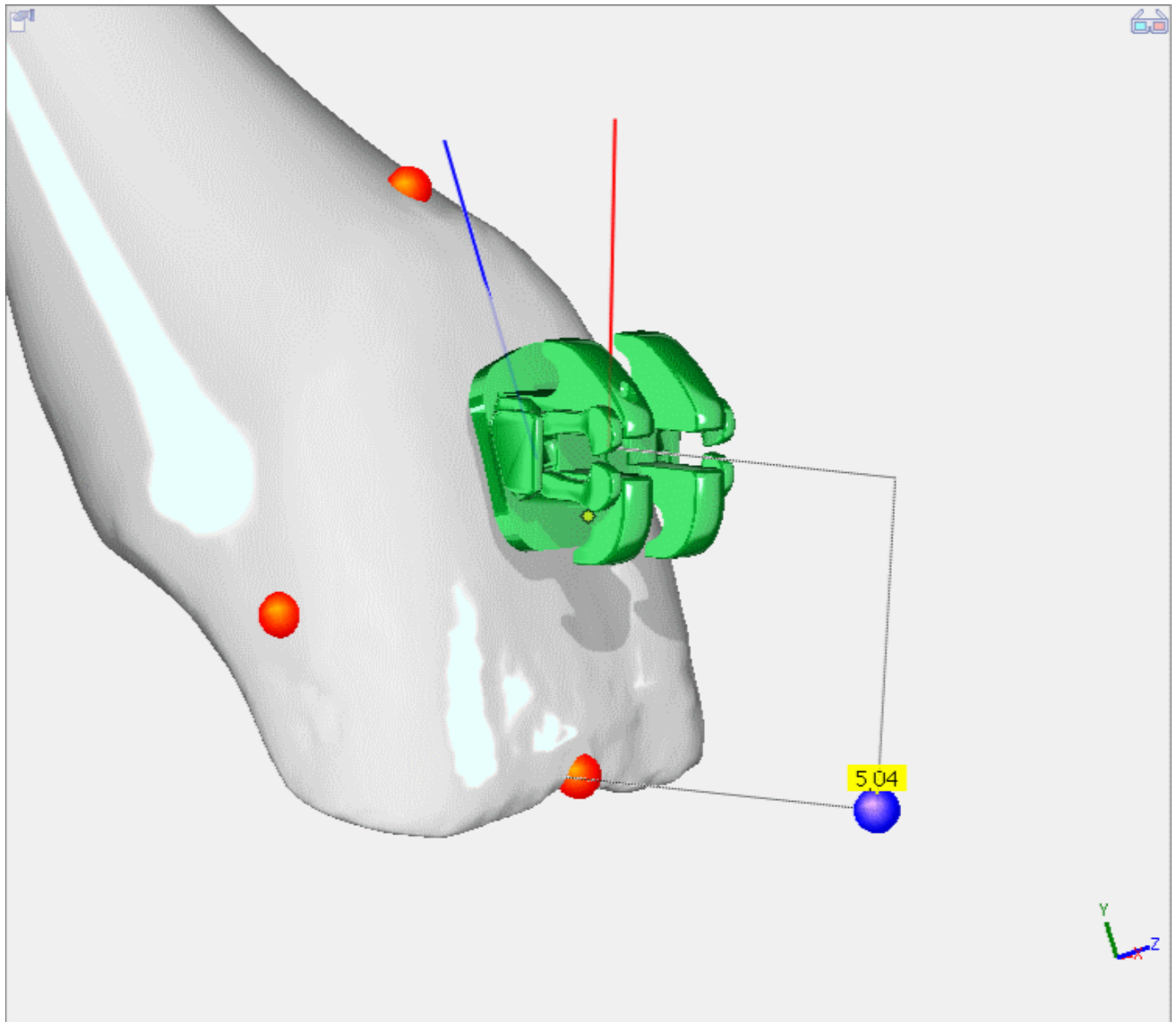

[back](#)

From:
<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:
http://onyxwiki.net/doku.php?id=en:fa_bl

Last update: **2023/10/30 08:04**



