

Main Window Tab |Forms|

Main tab |Forms| allows recording and managing of image-related information in the form of individual questionnaires. The layout of the forms has to be generated by means of external DHML editors by the user himself.

User Interface

The screenshot displays the 'Modellanalyse' (Model Analysis) form within the OnyxCeph 3D Lab software. The form is organized into several sections:

- DENTALSTADIUM:** Includes a dropdown menu for 'DENTALSTADIUM' and a section for 'Zahnwanderung' (Tooth Migration) with 'keine' (none) selected for both 'Oberkiefer' (Upper Jaw) and 'Unterkiefer' (Lower Jaw).
- OKKLUSALBEFUND SAGITTAL:** Contains a table for sagittal occlusal findings with columns for '1. Molar', 'Eckzahn' (Canine), and 'Front (Overjet)'. The 'Front (Overjet)' is set to '+9'.
- OKKLUSALBEFUND VERTIKAL:** Contains a table for vertical occlusal findings with columns for 'Lateral' and 'Front (Overbite)'. The 'Front (Overbite)' is set to 'normal'.

The interface also features a patient selection dropdown at the top, a date field (24.07.2014), and a status bar at the bottom indicating the workstation user (User [RK8]) and server (Server: ONYXDE3-[DAVID-LG]@DAVID-LG).

DHTML Templates

Even since OnyxCeph™ primarily being an imaging application, an effective workflow may also require recording, managing and displaying of specific not image-based patient data within the program. For these purposes DHTML forms generated by external editors can be imported and applied in the |Forms| index card. The form drafts can be stored in the categories

- Medical History
- Functional Diagnostics
- Clinical Documentation
- Others

Active form templates can be selected and called by the icon button [New] of the Forms dialog via Category and Form layout name.

The selected form is displayed, completed and saved under selectably name by the [Save Data] button within

the browser window. Already saved (completed) forms can be displayed and edited by the icon button [Open]. In addition the saved forms appear in the document tree on the |Patient| tab and, similar to all other entries, can be directly called there.

DHTML forms are displayed in the workspace of the |Forms| tab by means of an internet-browser window. The IE functionality used is adjusted to the version installed on your computer. Via context menu browser-specific calls are available.

From:

<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:

http://onyxwiki.net/doku.php?id=en:maintab_forms&rev=1588329052

Last update: **2020/05/01 12:30**

