


## Main Window Tab |Forms|

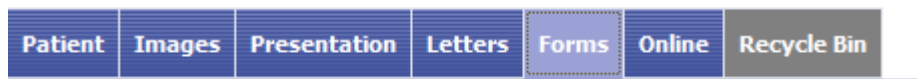

Main tab |Forms| allows recording and managing of image-related information in the form of individual questionnaires. The layout of the forms has to be generated by means of external DHML editors by the user himself.

## User Interface

OnyxCeph™ 3D Lab - Demo, Daniela - 10000

Practice Name: Image Instruments Patient: 10000: Demo, Daniela

Medical History: test1 (Saved) 24.07.2014

### Modellanalyse

DENTALSTADIUM: ---

Oberkiefer: groß, ausreichend, zu breit, jede Menge, keine

Unterkiefer: ausreichend, zu breit, dto., keine

OKKUSALBEFUND SAGITTAL

	1. Molar	Eckzahn	Front (Overjet)
rechts	2.00 Pb distal	2.00 Pb distal	+9
links	1.75 Pb distal	1.25 Pb distal	

OKKUSALBEFUND VERTIKAL

	Lateral	Front (Overbite)
rechts	normal	0
links	normal	0

Workstation: User [RK8] Server: ONYXD63-[DAVID-LG]@DAVID-LG

## DHTML Templates

Even since OnyxCeph™ primarily being an imaging application, an effective workflow may also require recording, managing and displaying of specific not image-based patient data within the program. For these purposes DHTML forms generated by external editors can be imported and applied in the |Forms| index card. The form drafts can be stored in the categories

- Medical History
- Functional Diagnostics
- Clinical Documentation
- Others

Active form templates can be selected and called by the icon button [New] of the Forms dialog via Category and Form layout name.

The selected form is displayed, completed and saved under selectably name by the [Save Data] button within the browser window. Already saved (completed) forms can be displayed and edited by the icon button [Open]. In addition the saved forms appear in the document tree on the [Patient] tab and, similar to all other entries, can be directly called there.

DHTML forms are displayed in the workspace of the [Forms] tab by means of an internet-browser window. The IE functionality used is adjusted to the version installed on your computer. Via context menu browser-specific calls are available.

From:

<http://onyxwiki.net/> - [OnyxCeph<sup>3</sup>™ Wiki]

Permanent link:

[http://onyxwiki.net/doku.php?id=en:maintab\\_forms&rev=1588333189](http://onyxwiki.net/doku.php?id=en:maintab_forms&rev=1588333189)

Last update: **2020/05/01 13:39**

