Main Window Tab |Forms|



Main tab |Forms| allows recording and managing of image-related information in the form of individual questionnaires. The layout of the forms has to be generated by means of external DHML editors by the user himself ¹⁾.

Main tab |Forms| was removed since release 3.2.169 due to potential cybersecurity treats which could result from browser script

User Interface

| OnyxCeph [™] 3D Lab - Demo, Daniela - 10000 | | | | | | | | | - | | x c |
|--|----------------|--|---------------------|-----|---------|--------|--------------------|-----------|--------------|----------|------------|
| Construction Name | | | | | | | License | Releas | se 📕 Suppo | rt 🕶 🖲 | 🗄 Languag |
| Practice Name Image Instruments | • 🔞 | Patient 10000: Demo, Da | aniela | - 2 | Patient | Images | Presentation | Letters | Forms On | ine Re | ecycle Bin |
| 🖻 💾 🔟 🗐 🔹 🚼 Hedical History: test1 (Saved) | | | | | | 7.2014 | | - | | | ð • C |
| Modellanalyse | | | | | | | | | | | ^ |
| DENTALSTADIUM V Oberkiefer Unterkiefer | | | | | | | | | | | |
| Apikale Basis | groß | ~ | ausreichend | ~ | | | | | | | |
| Kieferbreite | zu bteit | ~ | zu bteit | ~ | | | | | | | |
| Stellungsfehler einzelner Zähne | jede Menge | ~ | dto. | ~ | | | | | | | |
| | | ~ | | ~ | | | | | | | |
| Zahnwanderung | kene | ~ | keine | ~ | | | | | | | |
| | | ~ | | ~ | | | | | | | |
| OKKLUSALBEFUND SAGITTAL | 1. Molar | Eck | zahn Front (Overjet |) | | | | | | | |
| rechts | 2 00 Db distal | 2.00 Pb dis | | | | | | | | | |
| links | | 2.00 Pb dis 1.25 Pb dis | | ~ | | | | | | | |
| | 1.1310 0.3.0 | - [| | | | | | | | | |
| OKKLUSALBEFUND VERTIKAL | 1 | ateral | Front (Overbite | ;) | | | | | | | |
| rechts | normal | • 0 | ▼ normal | ~ | | | | | | | |
| links | | | ▼ normal | ~ | | | | | | | |
| | | | | _ | | | | | | | |
| | | | | | | 16 | | | | | _ |
| | | | | | | Wor | kstation: User [RK | 8] Server | : ONYXDB3-[D | AVID-LG] | BDAVID-LO |

DHTML Templates

Even since OnyxCeph[™] primarily being an imaging application, an effective workflow may also require recording, managing and displaying of specific not image-based patient data within the program. For these purposes DHTML forms generated by external editors can be imported and applied in the |Forms| index card.

The form drafts can be stored in the categories

- Medical History
- Functional Diagnostics
- Clinical Documentation
- Others

Active form templates can be selected and called by the icon button [New] of the Forms dialog via Category and Form layout name.

The selected form is displayed, completed and saved under selectably name by the [Save Data] button within the browser window. Already saved (completed) forms can be displayed and edited by the icon button [Open]. In addition the saved forms appear in the document tree on the |Patient| tab and, similar to all other entries, can be directly called there.

DHTML forms are displayed in the workspace of the |Forms| tab by means of an internet-browser window. The IE functionality used is adjusted to the version installed on your computer. Via context menu browser-specific calls are available.

1) Info

> From: http://onyxwiki.net/ - [OnyxCeph³™ Wiki]

Permanent link: http://onyxwiki.net/doku.php?id=en:maintab_forms&rev=1676449593



Last update: 2023/02/15 09:26