


Ceph Tracing

◀ TP Key Users


1. Open Tracing Module from Thumbnail

Click on main menu icon [Digitize] to open tracing module window.

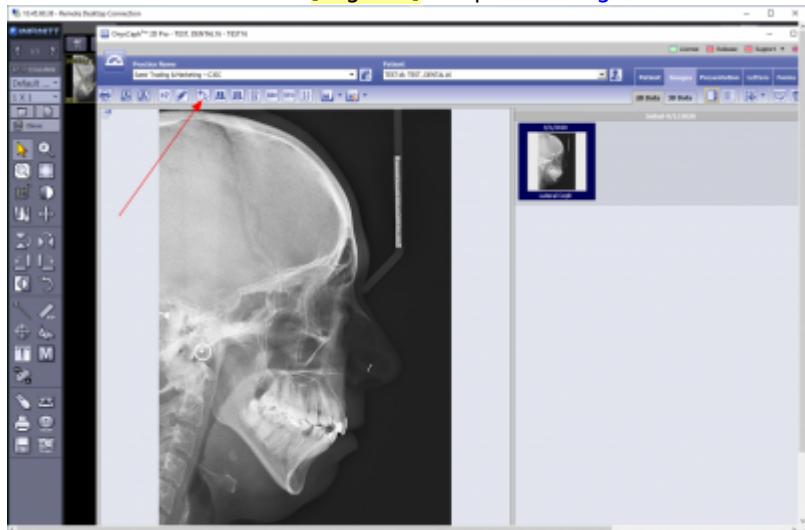


2. Select Analysis

Make sure [only] lateral ceph analysis [Riyahd] is selected - otherwise, [select analysis from Analysis List](#)
Make sure [image calibration](#) is correct.

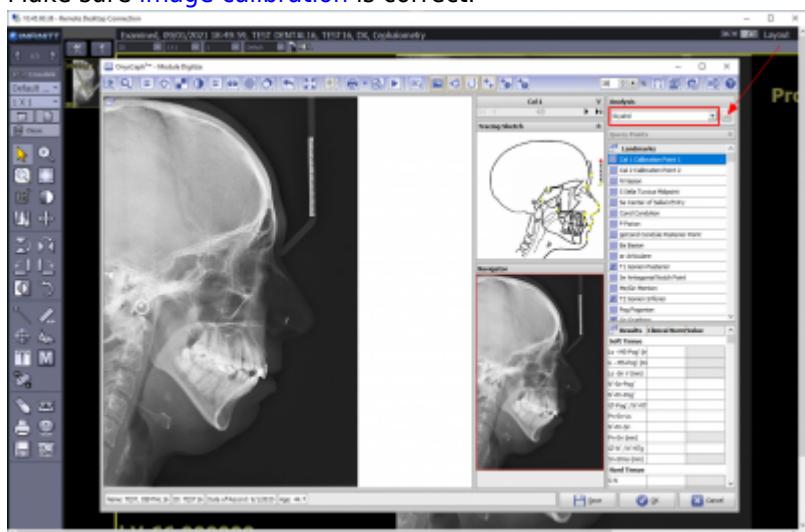


3. Digitize Ceph Landmarks

Digitize landmarks in landmark list by right clicking the related position in the image.
Correct landmark position by left mouse drag&drop.

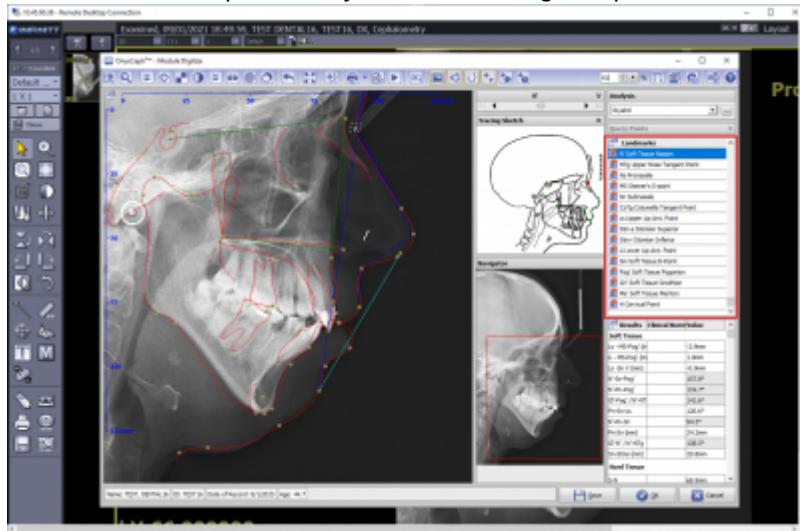


4. Adjust Profile Template

Optionally, adjust profile template by Bezier curve.

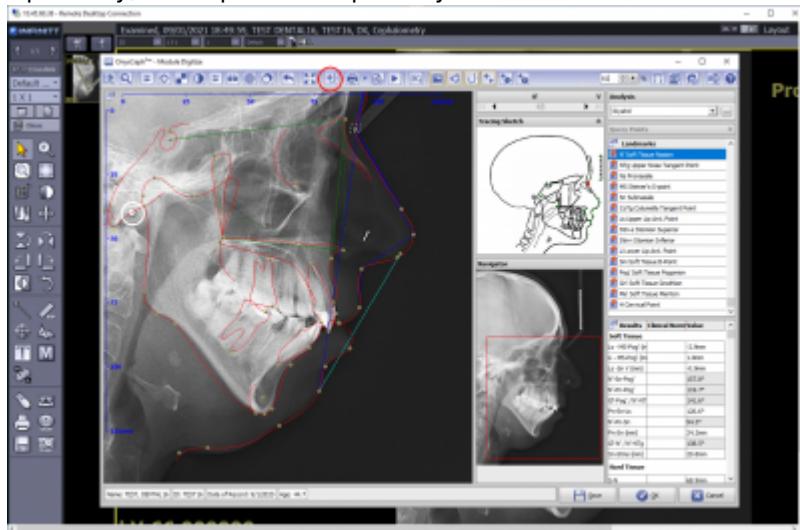

5. View Results

View results in [Result List] - open full screen list via context menu.

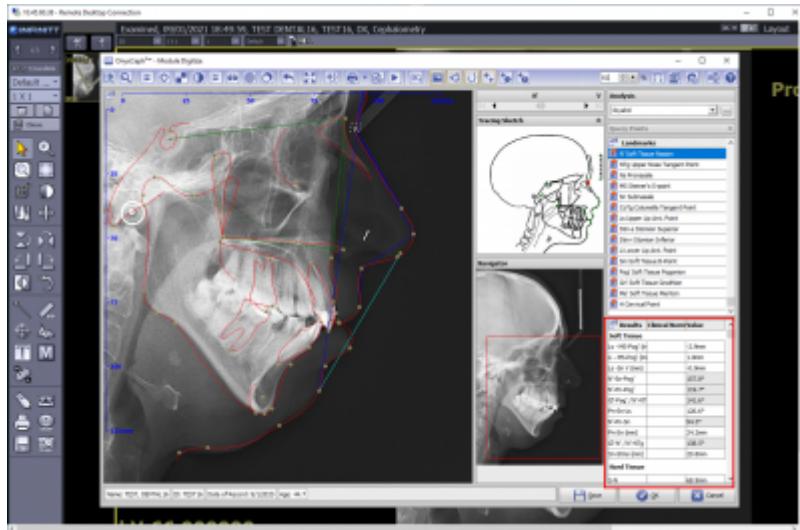

Links

 [Module Handout Digitize 2D](#)

[◀ TP Key Users](#)

From:
<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:
http://onyxwiki.net/doku.php?id=en:moh:tp_keyuser3

Last update: **2022/03/23 15:17**

