


Use Multiple Attachments per Crown

In module **Aligner 3D**, an attachment from the internal attachment library can be assigned to each crown via the context menu in the [Attachments] panel. If multiple attachments are to be used for one crown, then an entry [New point] can be created via the context menu in the [Attachments] panel and placed on the relevant crown by right-clicking. This changes the name of the entry from [New point] to the relevant tooth number (e.g. [Tooth 11]) and another attachment from the internal attachment library can be assigned to the same crown.

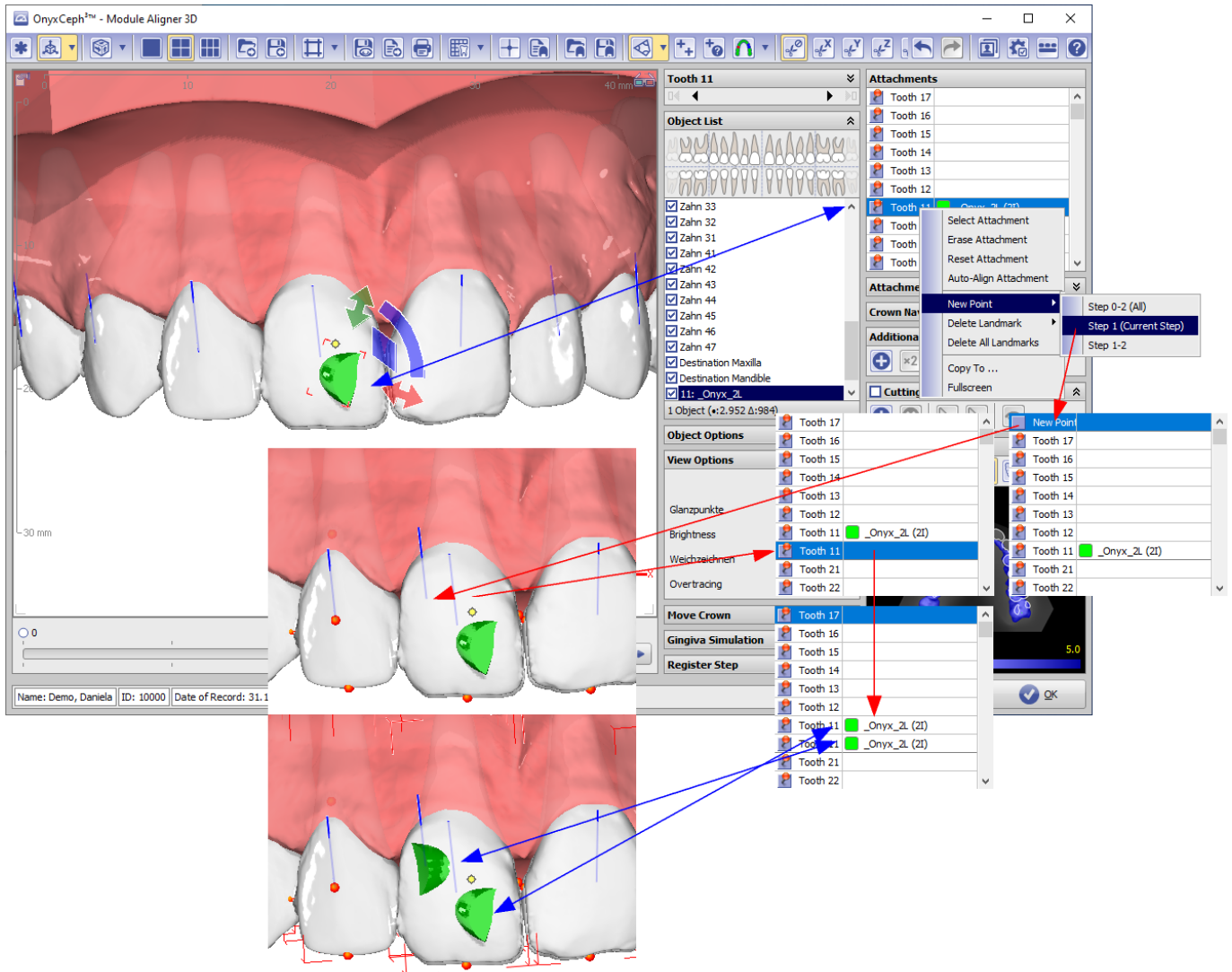

Option **AttachmnegrRuleEditor** can also be used to place multiple attachments on the same crown.

From:

<https://www.onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:

<https://www.onyxwiki.net/doku.php?id=en:multiattachments>

Last update: **2021/07/29 13:33**

