

Client Application OnyxReport



In order to make the information available in OnyxCeph³™ to people who do not have a software license themselves, reports can be compiled within the main program and saved locally as a file or uploaded to an FTP or web server.

User interface

The screenshot shows the 'Report' window of the OnyxReport application. The window is divided into several sections:

- Patient:** Contains a list of patient records. The first record is 'FB2354 - Roots, Tooth' and the second is '4178 - Klüber, Johannes'. There are icons for adding, deleting, and refreshing the list.
- Daten:** A section with a checkbox labeled 'Send Anonymously' which is checked.
- Options:** Contains a dropdown menu for 'Recipient's language' set to 'English', a checkbox for 'Password' which is unchecked, and a checkbox for 'Send Email with FTP-Upload' which is checked. Below these is a logo for 'SAMPERMANS IOS INTERNATIONAL ORTHODONTIC SEMINARS'.
- Images:** A table with columns 'No.', 'Type', 'Date of Record', and 'Image'. It lists four items: 1. CT-HeadScan (03.03.2022), 2. Lateral Right (03.11.2021), 3. Cast Deciduous 3D (23.11.2021), and 4. Cast Mixed 3D (23.11.2021). Each item has a checked checkbox in the 'Image' column. To the right of the table is a 3D rendering of a dental arch.
- Slides:** A tab next to 'Images'.
- Comment about Patient:** A text area with the placeholder 'Add patient specific comments here'.
- Sender:** A section with a sub-header 'Image Instruments' and a text area for 'Message for recipient' with the placeholder 'Add a message for the recipient here'.

At the bottom of the window are buttons for 'Help', 'Preview', and 'Close', along with standard window control icons (minimize, maximize, close) in the top right corner.

Links



From:

<https://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:

https://onyxwiki.net/doku.php?id=en:program_onyxreport

Last update: **2023/02/16 22:24**

