

# Interface TRIOS Scanner

---

## Patient Data (PMS)

In addition to the command line interface, the VDDS (PMS) interface for the Dental Desktop software is also supported as of release 1.7.19.1. This allows (only) the transfer of patient data to Dental Desktop. The same applies to the Unite online portal.

The settings are described [here](#) - regarding available updates please contact 3shape directly.

## Scan Data (IMS)

Since Q4 2022 there is a joint development project with 3shape to enable a direct import of scans to the patient in OnyxCeph<sup>3</sup>™ from Dental Desktop or from the online platform Unite. Unfortunately, this development is currently not yet complete, so that TRIOS scans still have to be exported manually to a patient directory. We recommend using the [simple image import interface](#) via bookmarks for this purpose.

From:

<http://onyxwiki.net/> - [OnyxCeph<sup>3</sup>™ Wiki]

Permanent link:

[http://onyxwiki.net/doku.php?id=en:trios\\_vdds](http://onyxwiki.net/doku.php?id=en:trios_vdds)

Last update: **2023/08/15 07:45**

