## Image Import images via Webcam

A simple way to import camera images taken with an integrated or USB connected webcam can be set up and used in System Options on tab |Third Party Tools|. For this purpose, enter the webcam application call under entry [3. start] (e.g. [microsoft.windows.camera:] for the Windows<sup>™</sup>-integrated camera app) and the destination folder used by the application for saving the captured images under entry [image Acquisition - Bookmarks] (e.g. [C:\Users\Image Instruments\Pictures\Camera Roll] for the Windows<sup>™</sup>-integrated camera app).

System Options					×
Patient Data	Practice Ma	anagement	Scanner XYZ	Windows Camera	carestre 🔹 🕨
Image Types	<ul> <li>✓ Item Visible Name: Windows Camera</li> <li>✓ Exit after a successful action</li> </ul>				
Sessions	Write entry in log file				
Galleries	Hand over Master Data Save Image Data				
Printouts	1. Bring Active Program to the Foreground				
Forms	2. Send Windows - Message				
Display	Image: Start Program / Open File         File:       microsoft.windows.camera:				
Interfaces	Parameter				
Third Party Tools	Parameters are used as file names				
	4. HTTP-Request				
Event	5. Send E-Mail				
Upload Data	Image Acquisition - Bookmarks				
Email	Directory	C <mark>:\Users\Imag</mark> Directories Aut	e Instruments\Pic omatically	tures\Camera Roll	•••Q
	File Mask				Q
Help Octose					

Then the camera app can be launched via menu [Go to  $\blacktriangleright$ ]



and the import of the captured images in the module Add Image 2D via the created camera bookmark.



From: https://onyxwiki.net/ - [OnyxCeph<sup>3</sup>™ Wiki]

Permanent link: https://onyxwiki.net/doku.php?id=en:webcam\_import

Last update: 2023/08/25 12:46

