# **Initial Server Configuration**

TP Onyx Admin



Note: This instruction is only needed if a new clinic server has to be installed

# 1. Create Onyx\_PDF Folder

Create folder C:\Onyx\_PDF on server

### 2. Import Patches

(1) Download patch archive MOH\_Onyx\_Patch.zip which includes

- Riyahd cephalometric analysis patch
- PDF print form template patch

File Download

(2) Install patches via menu item Options|Install Patch.

# 3. Set PDF Export Folder

(1) Open main menu (F10)

- (2) Select menu entry [Options|Systemoptions] on tab |Interfaces|
- (3) Click button [Directories]
- (4) Select ↓
  - Type: PDF
  - Checkbox [Active]: 🗸
  - Checkbox [Show Dialog]: 🗸
  - File name: C:\PDF\_Export\%1\_%X
  - Overwrite Document: Overwrite Existing

Export - Directories			×
Туре	PDF		<b>_</b>
Active			
Show Dialog			
File Name	C:\PDF_Export\%1_%X		•••
Preview	C:\PDF_Export\10000_20220309092503.PDF		
Overwrite Document?	Overwrite Existing		-
		🕑 ок	Cancel

#### 4. Select Analysis

- (1) Open Module Digitize on main tab |Images / 2D Data|
- (2) Open the Select Analysis window by button [...]
- (3) Unselect all options and analyses by button [None]
- (4) Then select only the Riyahd in analysis list.

## 5. Adjust Image Type Settings

(1) Open main menu (F10)

- (2) Select menu entry [Options|Systemoptions] on tab |2D Image Types|
- (3) Select Image Type [Lateral Ceph] and edit settings  $\downarrow$ 
  - Image Aspect Ratio (%): 64,84 [may be different for depending on PACS xray A/R]
  - Select box [View]: Riyahd
  - Select box [Printing Form]: KSA MOH Ceph

2024/03/31 09:08

3/4

Edit: Image Type	×			
General	*			
Image Type Name:	Lateral Ceph			
Image Type Category:	X-Ray			
Token:	XRCR			
Thumbnail:	Load			
	Delete			
Image Acquisition	*			
Maximum Image Size:	4096			
Image Aspect Ratio (%):	64,84			
Horizontal Guideline (%):	40▼			
Vertical Guideline (%):	80			
Rotate Image:	0°  Standard			
Fit Image to Frame:	Fit All			
Mirror Image Horizontally				
Tracing	*			
Tracing Type:				
Ceph Lateral				
Trace as Flipped				
Analyzing Methods				
Slide View	*			
View:	Printing Form:			
Riyahd	KSA MOH Ceph			
Pelp	OK Cancel			

# 6. Create Global Template

- (1) Open context menu on main menu button by [SHIFT]+[CTRL} + Right click
- (2) Click menu entry Configurations
- (3) Select and define local configuration [As global template] for subsequent installed clients

Onyx Configurations		×
View Configurations	;	
• Workstation: 'RK8'		
O Login: 'USER'		
O All Configurations		
O All Templates		
Configuration		
USER [RK8]		^
		*
Delete	As Template Apply Edit	
As Template for	Computer:	
		=
	As global template	
Section	Entry Value	
1	]	
Propagate Entry	Propagate File	

#### TP Onyx Admin

From: http://onyxwiki.net/ - [OnyxCeph<sup>3™</sup> Wiki]

Permanent link: http://onyxwiki.net/doku.php?id=moh:tp\_admin2

Last update: 2022/03/15 12:41

