Befundtransfer

Problem

Bei Patienten A [unter Praxisname X] befinden sich Bildbefunde, die zu Patient B [unte Praxisname Y] verschoben werden sollen.

Lösung

In Patientendatensatz A [Praxisname X]:

- Alle zu verschiebenden Thumbnails auf der Registerkarte |Bilder / 3D-Daten| auswählen
- Kontextmenü auf einem der ausgewählten Thumbnails öffnen
- Eintrag [Senden an Befundtransfer] auswählen



In Patientendatensatz B [Praxisname Y]:

- Einfügen der ausgewählten Befunde mit [STRG]+T oder über Menü [Patient|Patient wählen|Befundtransfer]
- Löschen der Originalbefunde in der falschen Patientenakte
- Falls erforderlich, wiederholen Sie den Arbeitsablauf für 3D-Bilder

workflow_0002

<u> </u>	nyxCeph³™ Lab - Ra	ey, Iri	n - 70054								-	• ×
								Notifications (1)	License	Release	Support	🛞 Language
4	Practice Nam	e		Patient								
	Patient	• 2	New Patient Ctrl+M	70054: Ray,	Irin		• 🙇	Patient Image	s Letters	Presentation	Forms	Recycle Bin
6	Edit	•	Select Patient	🔝 Card File	Ctrl+P							- D
	View	• 🗌	Delete Patient	Quick Ceph	Ctrl+Q			2D Data 30	Data		1 1 1 2	
	Communication	•	New Image	PACS	Shift+Ctrl+D	s-11.05.2009						8
	<u>G</u> o To	۰ 🗖	Celect Practice Chiff (Ctrl)	Ending-Transfer	Ctrl+T	s-25.04.2010						
	Options	비법	g allectriacace anti-curr	dentaleyepad	Shift+Ctrl+E	12.08.2015						
=	Help	۰ŀ			Observat	ion-14.07.2022						
=	Egit	-		Initial-26.09.2022								
20:	4 30:8	-			Observation-26.09.2022							
2D:	5 3D: 32			/	Observation-27.09.2022							
	21.12.2022		21.12.2022	21.12.2022	21.12.2		21.12.2022					
	Anterior Occlusion		Right Occlusion	Left Occlusion	Upper Oct	dusal	Lower Occlusal					
20:	0 3D: 0	-		Observation-13.10.2022								
20:	4 30: 0 30. 12. 2022		30.12.2022	30. 12. 2022	30.12.21 Oblique I	-eft						
Nam	: Ray, Irin ID: 7005	4 B	thday: 28.09.2012 Age: Created:	Changed:				104	Workstation: L	Jser [RK8]	ver: ONYXD	33-[D61] @D61

From: http://onyxwiki.net/ - [OnyxCeph^{3™} Wiki]

Permanent link: http://onyxwiki.net/doku.php?id=workflow_0002

Last update: 2023/06/02 13:18

