

#3531 Adjust Cutting Line

For Releases > 3.2.35 in Modul [Cast Adjust](#) the polyline which is used to clip/cut the scan data to add a standard base tray can be adjusted afterwards. For this purpose, the node to be moved has to be selected again and then placed by a rightmouse click onto its new position. Also in case the line was closed already (green line), by [CTRL]+Z the edit mode can be reopened.

From:

<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:

<http://onyxwiki.net/doku.php?id=en:3531>

Last update: **2015/10/27 12:38**

