

Module Add Image 2D



Program version	BASIC	2D PRO	3D PRO	LAB	OMS
Availability	✗	✖	✖	✖	✖


✗ not included
 ✖ included by
 default
 ⓘ optionally
 included

In module Add Image 2D, 2D data (photos, xrays) can be imported from various sources. Record dates, [sessions](#) and [image types](#) can be assigned, [image attributes and comments](#) can be added and finally, the image can be saved in the Onyx database. Images can be imported individually or in blocks (batch import) and the appropriate patient can also be selected automatically using the QR code option.

First Steps

1. Activate "Clipping (Line)" in tab "Clip Object"
2. Place Landmarks at the model border, last point close to the first one
3. Press button "Trim" and confirm the correct trimming
4. Select and add socket in tab "Model Base"
5. Save as new finding

Module user interface

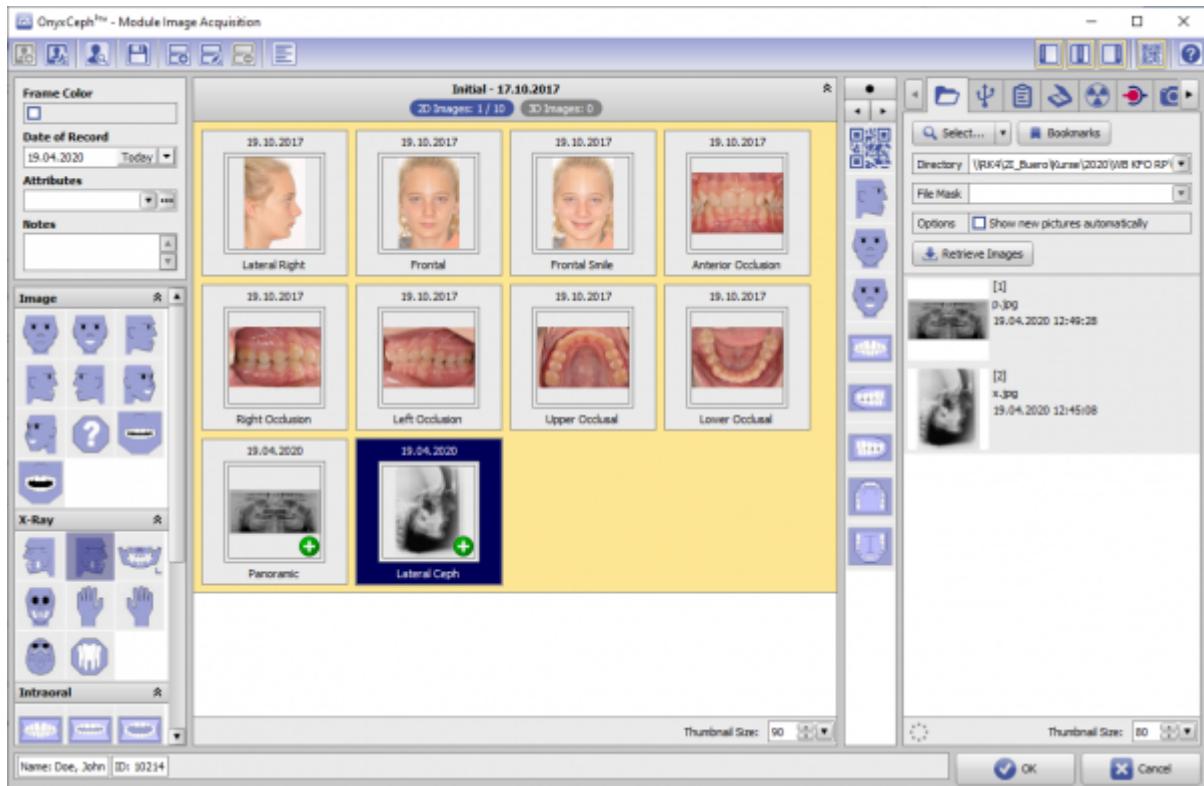

Links

 [Module Handout Image Import 2D](#)

From:
<https://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:
https://onyxwiki.net/doku.php?id=en:add_image_2d

Last update: **2023/03/06 12:45**

