

Module Cast Adjust



Program version	BASIC	2D PRO	3D PRO	LAB	OMS
Availability					


not included
 included by default
 optionally included

Module Cast Adjust allows to re-align the jaw relation of a scan according to different specifications and the attachment of different types of model bases (EU, US, plate, horseshoe bases) after manual definition of a trim line. The study model generated in this way must be saved as a separate finding.

First Steps

1. Open Module with scan (open shell - two arches in occlusion)
2. Optional: Adjust jaw relation on tab |Articulate|, |Jaw Rotation Axis| or |Align Jaw|
3. On tab |Clip Object| by right clicks, define trim line for upper and lower jaw
4. Clip along trim line Press
5. On tab |Model Base| select type of base tray and merge with scan
6. Save study model as new finding

Module user interface

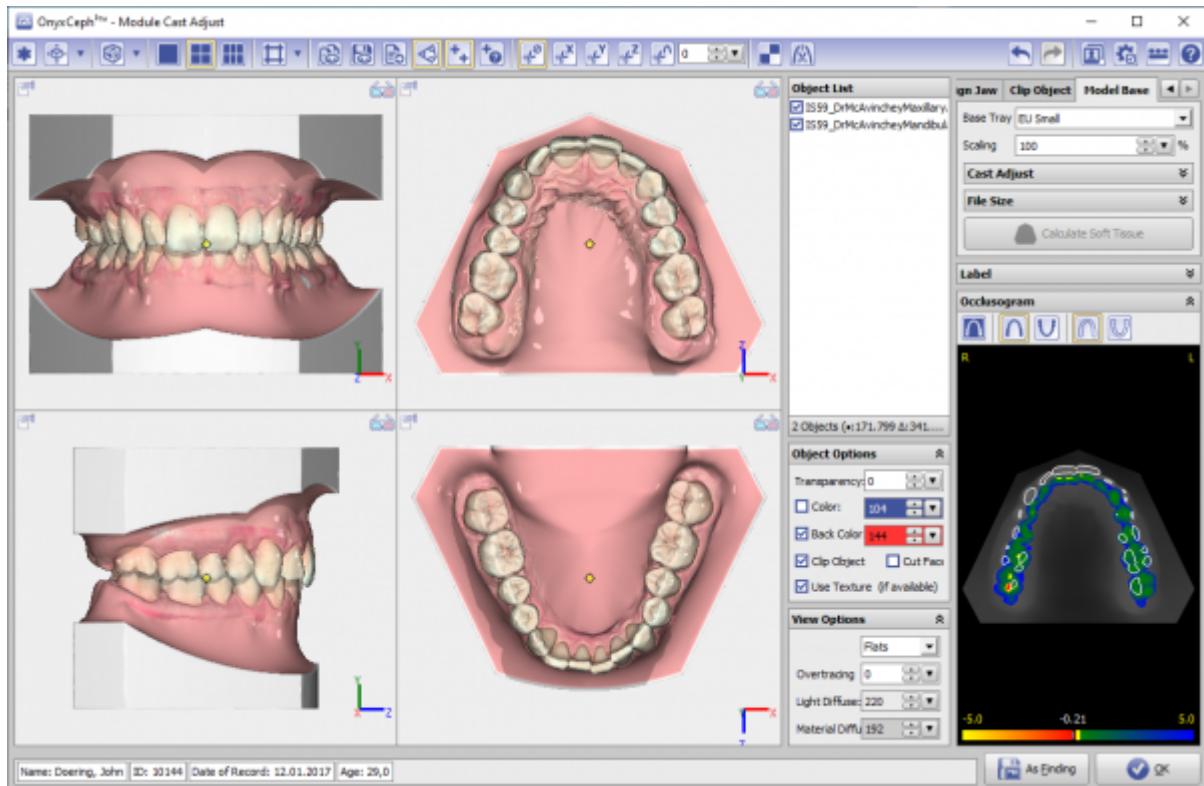

Links

-  [Module Handout Cast Adjust](#)
-  <https://youtu.be/TAAGkmFIKD4>
-  <https://drive.google.com/open?id=0B5nAlDRchl5WakJBXJ2QWNFSXM>

From:

<https://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:

https://onyxwiki.net/doku.php?id=en:base_tray&rev=1678371443

Last update: **2023/03/09 15:17**

