



Import Carestream Landmarks

Carestream Dental Imaging offers option [Autotracing] for acquired lateral ceph images, which can be used to automatically find certain reference points.

If this option is activated, a file H*_tracing.xml with the reference point information is stored side-by-side to the original image H*.ceph in the predefined patient image folder and can be taken over to OnyxCeph^{3™} in parallel to the import of the lateral ceph converted by Carestream 8Bit stored in subfolder ..\JPX.

If the Carestream images are imported via predefined bookmarks ([simple image import interface](#)), the file format *.ceph must be allowed in addition to the *.jpg format when using file masks in order to allow finding the XML tracing file associated with the lateral ceph.

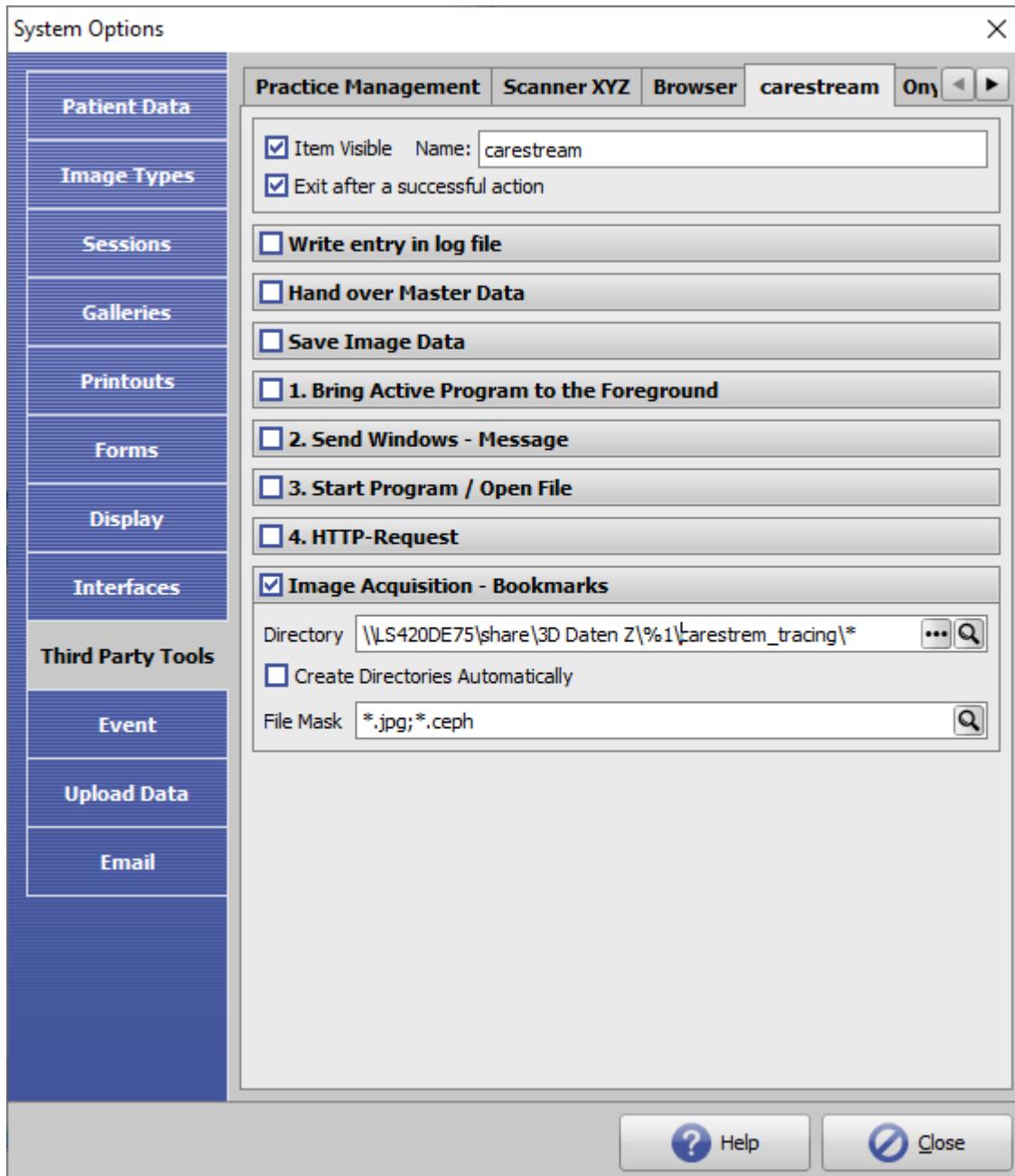

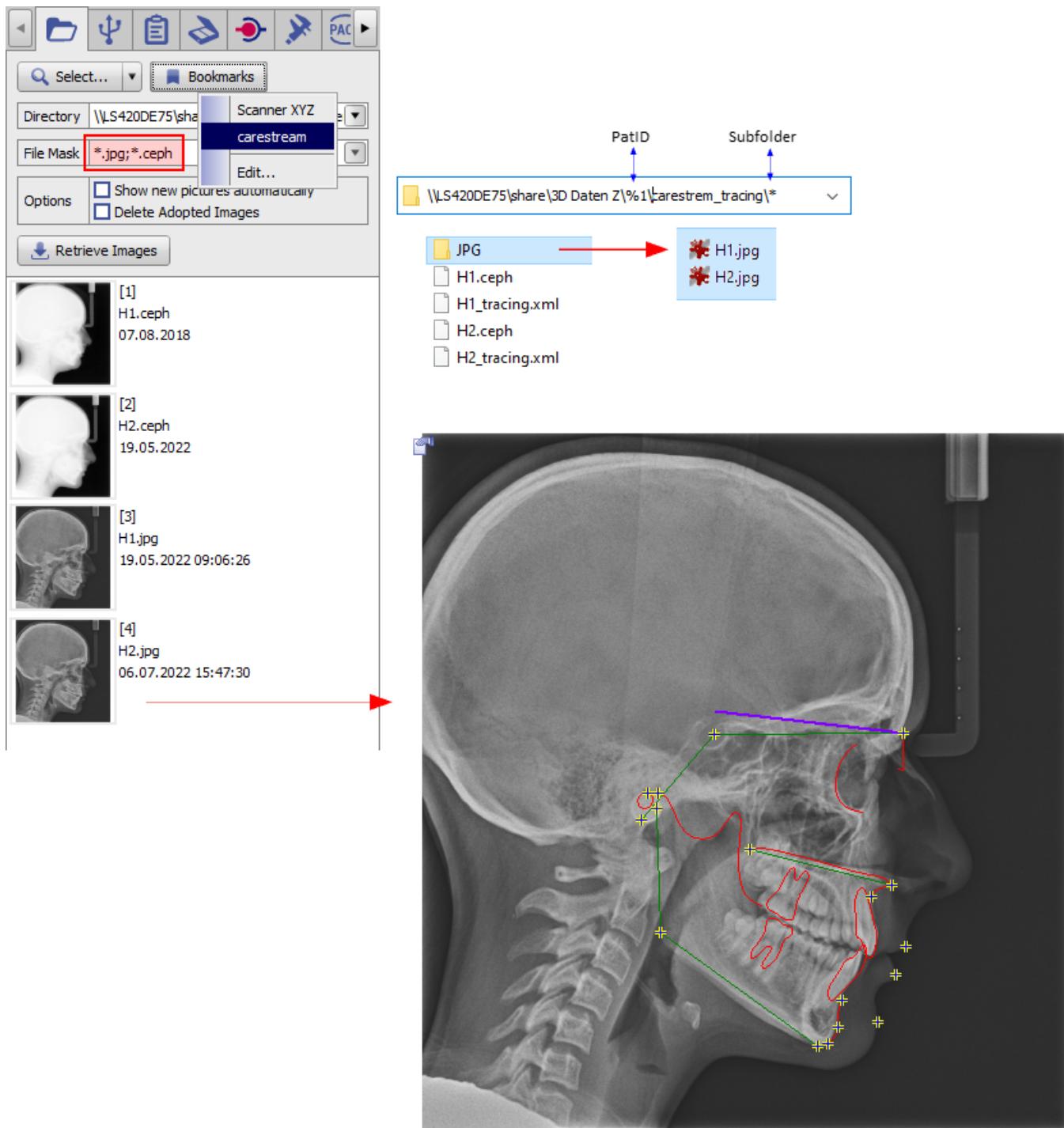

From:
<http://onyxwiki.net/> - [OnyxCeph³™ Wiki]

Permanent link:
http://onyxwiki.net/doku.php?id=en:cs_tracing

Last update: **2022/07/07 15:08**

