

# Image Type List 2D

Image Type List 2D [Image Type List 3D](#)


In OnyxCeph<sup>3™</sup> several pre-defined 2D image types can be used to classify imported patient images. By classifying the imported image with an image type, specific properties are assigned to the imported image which can be useful for its further processing in the software. All available 2D image types are assigned to categories

- Image
- X-Ray
- Intraoral
- Model



and can be activated, deactivated and customized by menu item [Options|System Options on Tab 2D Image Types](#) .

The image type **Code** can be used in the Command Line Interface (CLI) to specify the type of images passed. In addition, the codes are used in OnyxQuery.

## Category Image

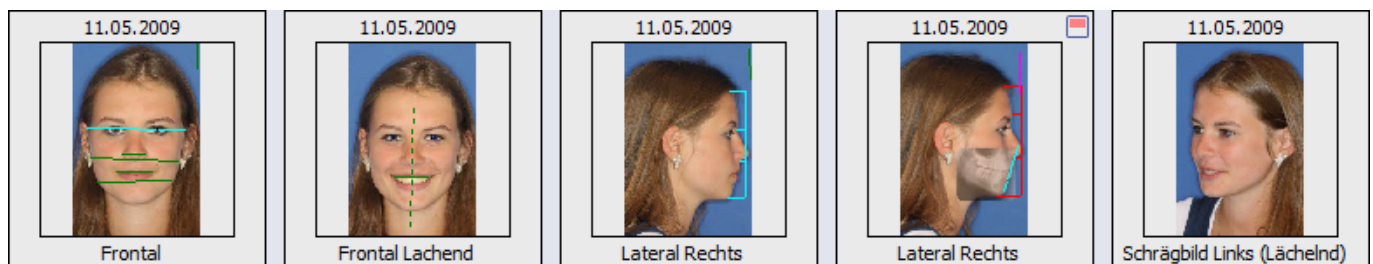


| Icon                                                                                | Name               | Name for Code         | Code |
|-------------------------------------------------------------------------------------|--------------------|-----------------------|------|
|  | Frontal            | PhotoFrontal          | PF   |
|  | Frontal Open       | PhotoFrontalOpen      | PFO  |
|  | Frontal Smile      | PhotoFrontalSmile     | PFS  |
|  | Lateral Right      | PhotoLateralRight     | PLR  |
|  | Lateral Open Right | PhotoLateralRightOpen | PLRO |
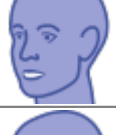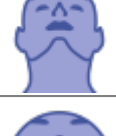
| Icon                                                                                | Name                | Name for Code          | Code  |
|-------------------------------------------------------------------------------------|---------------------|------------------------|-------|
|    | Lateral Open Smile  | PhotoLateralRightSmile | PLRS  |
|    | Lateral Left        | PhotoLateralLeft       | PLL   |
|    | Lateral Open Left   | PhotoLateralLeftOpen   | PLLO  |
|    | Lateral Open Smile  | PhotoLateralLeftSmile  | PLLS  |
|    | Oblique Right       | PhotoObliqueRight      | POR   |
|    | Oblique Open Right  | PhotoObliqueRightOpen  | PORO  |
|   | Oblique Smile Right | PhotoObliqueRightSmile | PORS  |
|  | Oblique Left        | PhotoObliqueLeft       | POL   |
|  | Oblique Open Left   | PhotoObliqueLeftOpen   | POLO  |
|  | Oblique Smile Left  | PhotoObliqueLeftSmile  | POLS  |
|  | No Type             | NoType 2D              | NO2D  |
|  | Bottom View         | PhotoFrontalSMV        | PFSMV |
|  | Nose Mouth          | PhotoFrontalNoseMouth  | PFNM  |
|  | Nose                | PhotoFrontalNose       | PFN   |

| Icon                                                                                | Name                    | Name for Code             | Code  |
|-------------------------------------------------------------------------------------|-------------------------|---------------------------|-------|
|    | Frontal Mouth Relaxed   | PhotoFrontalMouthRelaxed  | PFMR  |
|    | Frontal Mouth Wide Open | PhotoFrontalMouthWideOpen | PFMWO |
|    | Crown                   | PhotoCrown                | PC    |
|    | User 1                  | USER1                     | US1   |
|    | User 2                  | USER2                     | US2   |
|    | User 3                  | USER3                     | US3   |
|   | User 4                  | USER4                     | US4   |
|  | User 5                  | USER5                     | US5   |
|  | User 6                  | USER6                     | US6   |
|  | User 7                  | USER7                     | US7   |

## Category X-Ray

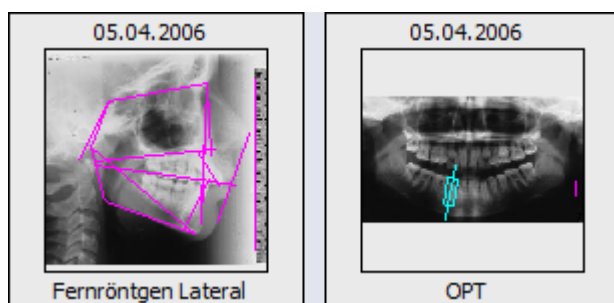

| Icon                                                                                | Name                 | Name for Code          | Code   |
|-------------------------------------------------------------------------------------|----------------------|------------------------|--------|
|    | Lateral Ceph (left)  | XRayCephalogramLeft    | XRCL   |
|    | Lateral Ceph         | XRayCephalogramRight   | XRCR   |
|    | Panoramic            | XRayPanoramic          | XPAN   |
|    | TMJ Left             | XRayTMJLeft            | XRTMJL |
|    | TMJ Right            | XRayTMJRight           | XRTMJR |
|    | TMJ                  | XRayTMJ                | XRTMJ  |
|   | PA Ceph              | XRayCephalogramPostAnt | XRCPA  |
|  | AP Ceph              | XRayCephalogramAntPost | XRCAP  |
|  | Left Hand            | XRayHandLeft           | XRHL   |
|  | Handröntgen (rechts) | XRayHandRight          | XRHR   |
|  | SMV                  | XRaySMV                | XRSMV  |
|  | Teeth                | XRayTeeth              | XRTT   |
|  | Tooth 1              | XRayTooth1             | XRT1   |
|  | Tooth 2              | XRayTooth2             | XRT2   |

| Icon                                                                              | Name    | Name for Code | Code |
|-----------------------------------------------------------------------------------|---------|---------------|------|
|  | Tooth 3 | XRayTooth3    | XRT3 |
|  | Tooth 4 | XRayTooth4    | XRT4 |
|  | Tooth 5 | XRayTooth5    | XRT5 |
|  | Tooth 6 | XRayTooth6    | XRT6 |
|  | Tooth 7 | XRayTooth7    | XRT7 |


## Category Intraoral



| Icon                                                                                | Name                         | Name for Code                      | Code   |
|-------------------------------------------------------------------------------------|------------------------------|------------------------------------|--------|
|  | Anterior Occlusion           | IntraOralOcclusionAnterior         | IOOA   |
|  | Anterior Occlusion Relaxed   | IntraOralOcclusionAnteriorRelaxed  | IOOAR  |
|  | Anterior Occlusion Wide Open | IntraOralOcclusionAnteriorWideOpen | IOOAWO |
|  | Right Occlusion              | IntraOralOcclusionRight            | IOOR   |
|  | Left Occlusion               | IntraOralOcclusionLeft             | IOOL   |

| Icon                                                                              | Name           | Name for Code              | Code    |
|-----------------------------------------------------------------------------------|----------------|----------------------------|---------|
|  | Upper Occlusal | IntraOralMaxillaOcclusal   | IOMAXO  |
|  | Lower Occlusal | IntraOralMandibulaOcclusal | IOMANDO |
|  | Overjet Right  | IntraOralOverjetRight      | IOOJR   |
|  | Overjet Left   | IntraOralOverjetLeft       | IOOJL   |

## Category Model

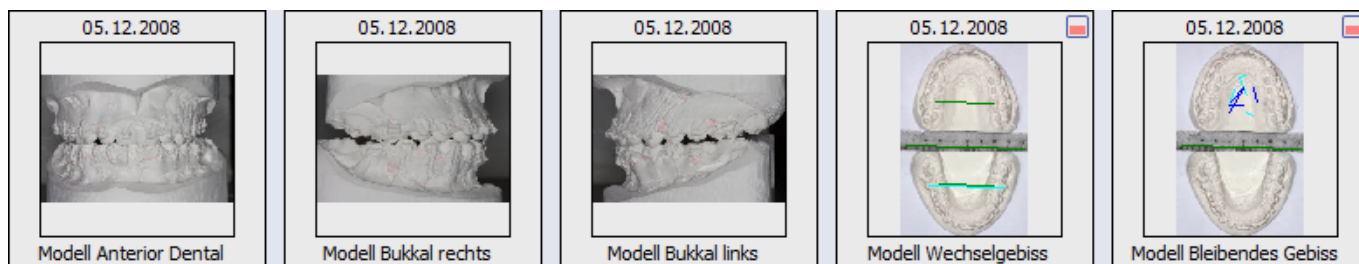

| Icon                                                                                | Name                 | Name for Code         | Code    |
|-------------------------------------------------------------------------------------|----------------------|-----------------------|---------|
|  | Cast Occlusion Upper | CAstMaxillaOcclusal   | CAMAXO  |
|  | Cast Occlusion Lower | CAstMandibulaOcclusal | CAMANDO |
|  | Cast Anterior        | CAstAntDental         | CAAD    |
|  | Cast Buccal right    | CAstBuccalRight       | CABR    |
|  | Cast Bukkal left     | CAstBuccalLeft        | CABL    |
|  | Cast Deciduous       | CAstDeciduous         | CAD     |

| Icon                                                                              | Name           | Name for Code | Code |
|-----------------------------------------------------------------------------------|----------------|---------------|------|
|  | Cast Mixed     | CAstMixed     | CAM  |
|  | Cast Permanent | CAstPermanent | CAP  |

From:

<https://onyxwiki.net/> - **[OnyxCeph<sup>3</sup>™ Wiki]**

Permanent link:

[https://onyxwiki.net/doku.php?id=en:imagetypes\\_2d](https://onyxwiki.net/doku.php?id=en:imagetypes_2d)

Last update: **2023/02/16 22:34**

