

Interface Settings CT|PACS|KIS: PACS

CT / PACS / KIS
✕

PACS

HL7



DIGITAL IMAGING AND COMMUNICATIONS IN MEDICINE

PACS - Support Activated

PACS Address

PACS Port

PACS AE-Title

Client AE-Title

Client Receiving Port

Patient ID:

Parameter	Setting
PACS - Support Activated	Activate PACS interface
PACS - Address	IP address or computer name of the PACS
PACS - Port	The port on which the PACS waits for requests
PACS - AE-Title	A pplication E ntity Title of the PACS
Client - AE-Title	A pplication E ntity Title of the OnyxCeph workstation
Client Receiving Port	The port on the client to which the PACS should send image data
Button [Echo]	<i>Function test:</i> An ECHO request is sent to the PACS
Patient ID:	<i>Function test:</i> Enter here the ID of a patient known in the PACS
Button [Query Patient]	<i>Function test:</i> Data of the patient with the ID are queried
Button [Query Image]	<i>Function test:</i> An image of the patient with the ID is requested and received.

From:

<https://www.onyxwiki.net/> - **[OnyxCeph³™ Wiki]**

Permanent link:

<https://www.onyxwiki.net/doku.php?id=en:pacs>



Last update: **2022/08/22 11:44**