

# Pulldown Menu Patient

The fastest way to open or change a patient record within the program is via the patient selection list. The 10 patient records last opened at the client (ID: last name, first name) are displayed and can be selected and reopened there.

Via the entry History at the end of the pull-down list, an overview of the patient records opened at the client station (or optionally at all client stations) within the previous 2 weeks can be displayed and used to re-open the patient record ► [read more](#)

OnyxCeph<sup>3™</sup> 3D Lab - Doering, John - 10144D

Practice Name: Image Instruments

Patient: 10144D: Doering, John

10000: Demo, fffff  
D-0006Neu: Hansen, Lisa  
10139: Dreher, F  
00007012: Dente  
Ignident\_2020: t  
Proaxis: Иркыт  
10152: Prco###  
10014: Doe, Joh  
22222: Patient, t  
History...

Patient's ID	Last Name	First Name	Gender	Model № / L...	Date				
10206	Dc		Female		12.12.				
5046@DICO...	Dr		Female		01.01.				
10139	Dr		Male		23.02.				
10043	Dr		Male		22.05.2005				3
10147	Dr		Male		28.01.2006				18
10108	ds		Male		16.05.2009				0
21212	Du		Male		04.07.2006				2
10109	Ed		Intersexual		12.12.1987				2
22787	EF		Female		24.01.1981				1
D-0005	Eik		Male		25.11.2005				0

From:

<https://www.onyxwiki.net/> - [OnyxCeph<sup>3™</sup> Wiki]

Permanent link:

[https://www.onyxwiki.net/doku.php?id=en:pulldownmenu\\_patient](https://www.onyxwiki.net/doku.php?id=en:pulldownmenu_patient)

Last update: 2021/02/13 20:05

