



Easy 3D Model Export in Occlusion

Steps

- (1) Select the dataset to be exported in the thumbnail overview
- (2) Right-click to open the thumbnail context menu

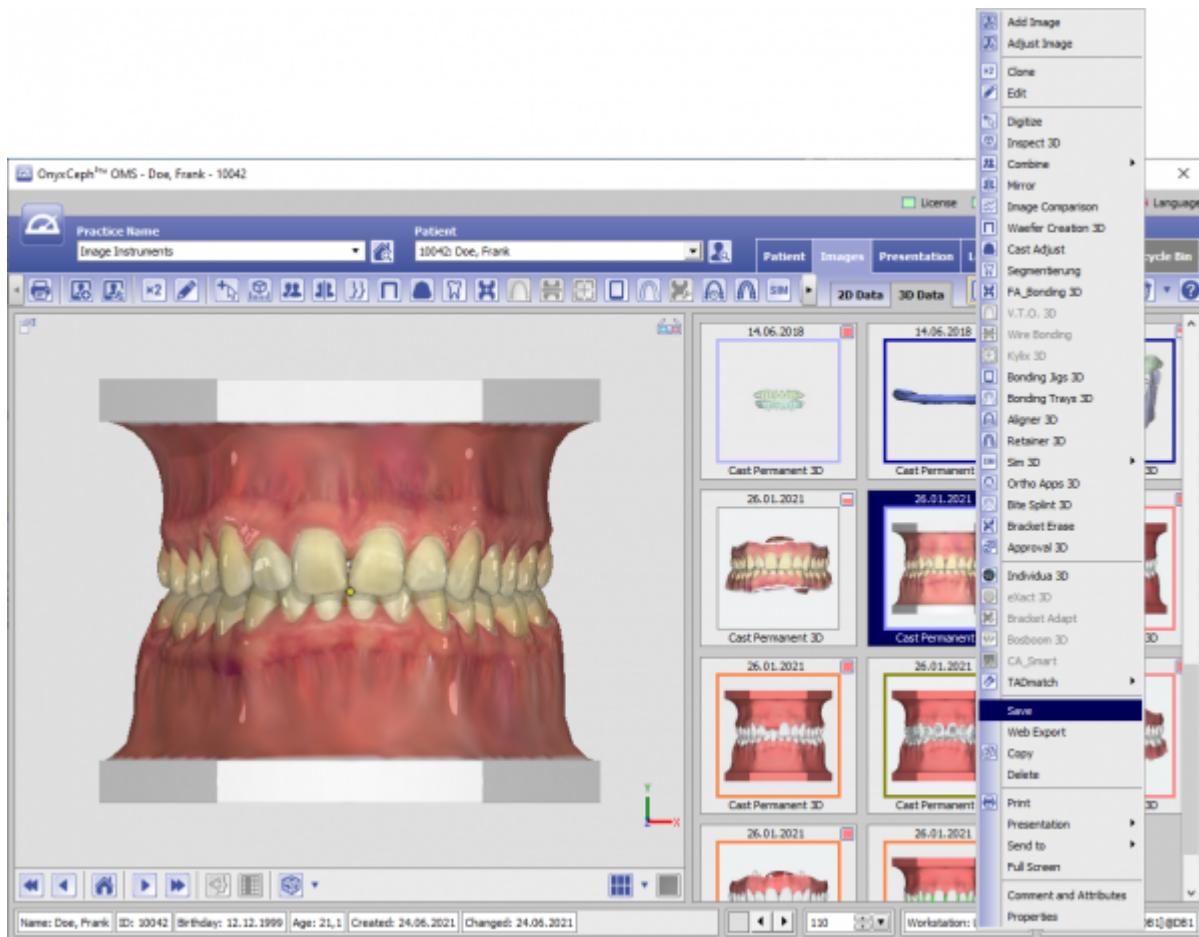

- (3) Select Eintrag [Save].

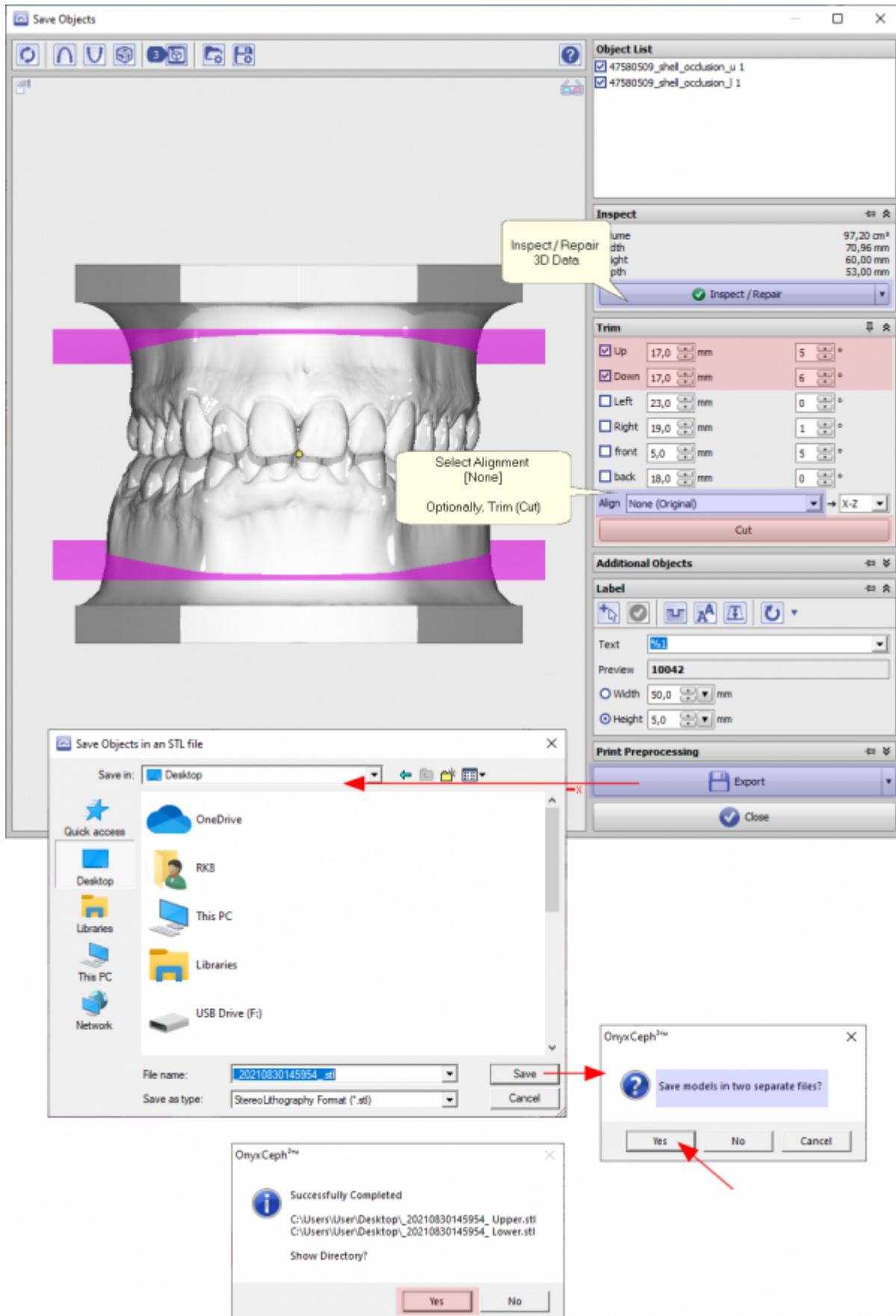

- (4) Check/repair the data set with regard to network errors
- (5) *Optional:* Define and use section planes
- (6) Ensure that the alignment of the OK and the mandible is not changed in order to maintain occlusion
- (7) *Optional:* Label the maxilla and/or mandible
- (8) Export the dataset to a selected directory

(9) *Optional:* Open the export directory in the explorer\ after saving

Links

 [Workflow 3D Export \(detailed\)](#)

From:
<http://onyxwiki.net/> - [OnyxCeph^{3™} Wiki]

Permanent link:
<http://onyxwiki.net/doku.php?id=en:simple3dexport>

Last update: **2021/08/30 16:42**

