

# System Options - Tab |Event|

On system options tab |Event|, reactions to internal application events can be configured.

System Options

Patient Data

Image Types

Sessions

Galleries

Printouts

Forms

Display

Interfaces

Third Party Tools

Event

Upload Data

Email

☐ Active

Event: 3D Export Done

☐ Query

☒ Exit after a successful action

☐ Write entry in log file

☐ Hand over Master Data

☐ 1. Bring Active Program to the Foreground

☐ 2. Send Windows - Message

☐ 3. Start Program / Open File

☐ 4. HTTP-Request

☐ 5. Send E-Mail

?

Help

Close

[Click on tab to switch to related topic]

## Events

Specifier	Occurrence
3D Export Done	after exporting a 3D dataset as .STL, .OBJ, etc.
3D Series Export Done	after 3D series export was finished etc.
Container Export Done	after creating a container

Container Import Done	after importing a container
Container Upload Done	after uploading a container to an FTP server
Evaluation Done	after the evaluation of a finding in the Digitize module
Image Import Done	after the transfer of a 3D or 3D dataset into the database
Module Closed	after a module has been closed
Module Opened	after a module has been opened
Project Saved	after the creation of a project
Report Export Done	after a report has been created and saved
Report Upload Done	after uploading a report to a server
Tooth Movement Saved	after saving the tooth movements in the Aligner module
Treated Image Saved	after saving a result As Findings
User Login	after a user has logged in
User Logout	after a user has logged out
Web Export Done	after exporting a 3D web view

In the events [Mergefields](#) are supported.

## Options

If the option **[X] Query** is active, the user is asked whether the actions for the event are to be carried out.

<a href="#">Write entry in log file</a>
<a href="#">Hand over Master Data</a>
1. Bing Active Program to the Foreground
2. Send Windows-Message
3. Start Program / Open File
4. <a href="#">HTTP Request</a>
5. Send email

If the option **[ ] Exit after a successful action** is not active, all active actions are executed.

If the option is active, processing is terminated starting with **1.** after the first active and successfully executed action.

From:

<http://onyxwiki.net/> - **[OnyxCeph<sup>3</sup>™ Wiki]**

Permanent link:

[http://onyxwiki.net/doku.php?id=en:tab\\_event](http://onyxwiki.net/doku.php?id=en:tab_event)

Last update: **2022/06/24 13:33**

