


# Tracing Options


Using the [Options] button in the panel, you can make various settings on individual tabs with regard to graphics overlays on 2D and 3D image windows.

## Tab |Variables|

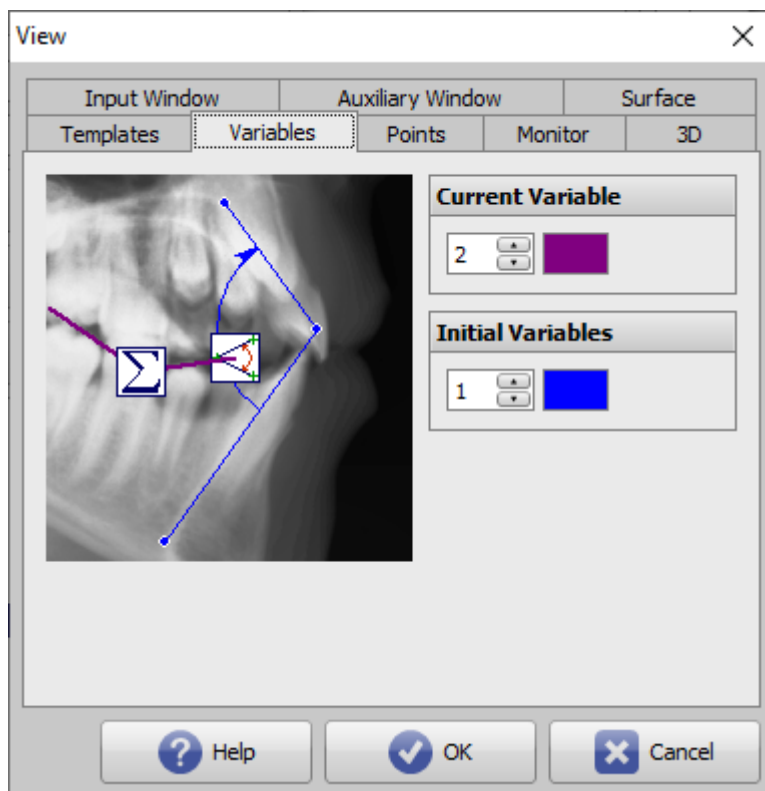

[Click on tab opens related wiki page]

In the context menu above a calculated value in the Results table in module Digitize, the definition used to calculate the value can be illustrated graphically.

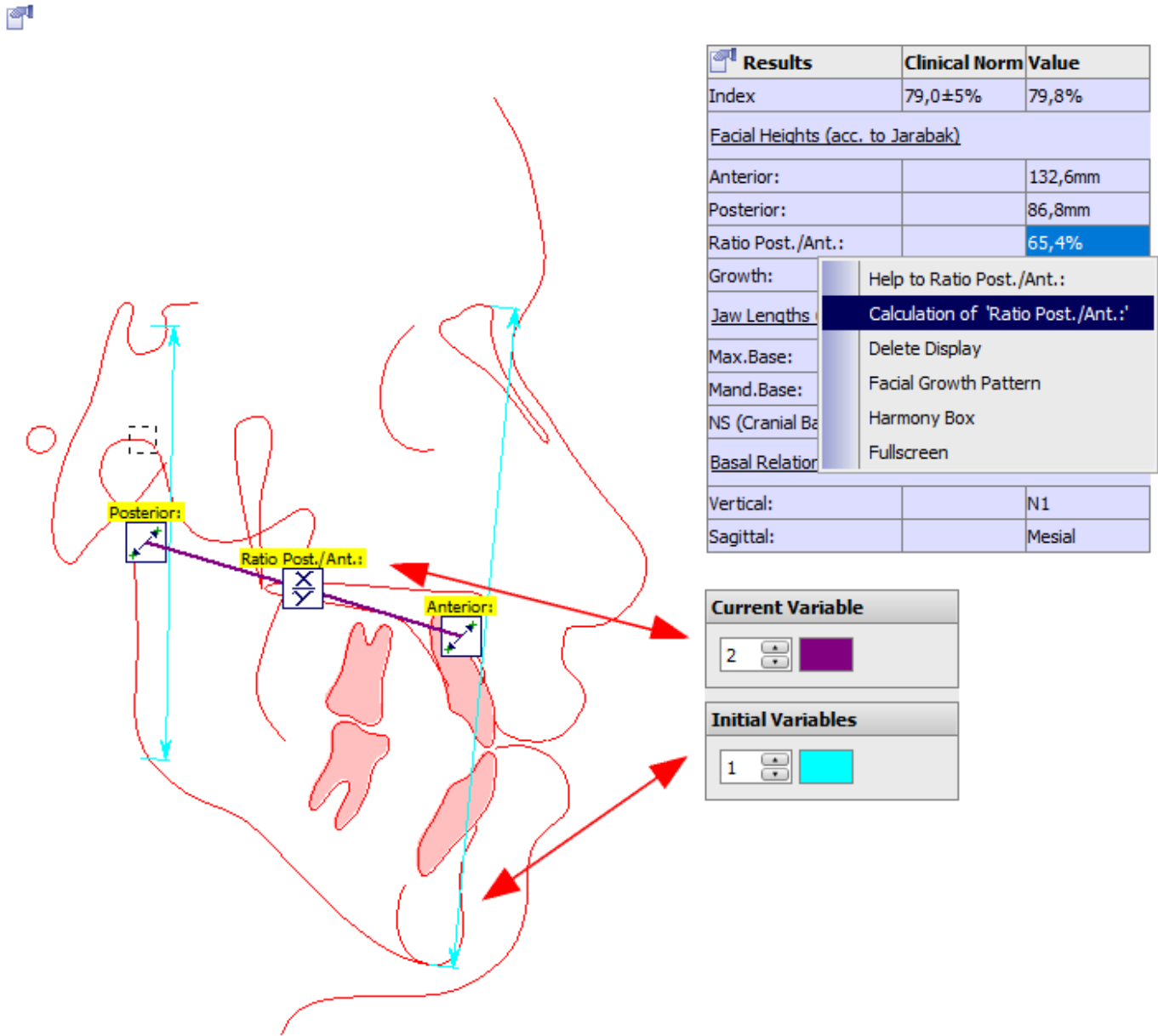


**Panel Current Variable**

The color and line width used for displaying the result variable can be specified here.

**Panel Initial Variables**

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The color and line width used for displaying the initial variables can be specified here.



From:  
<https://onyxwiki.net/> - [OnyxCeph<sup>3</sup>™ Wiki]

Permanent link:  
[https://onyxwiki.net/doku.php?id=en:to\\_variables](https://onyxwiki.net/doku.php?id=en:to_variables)

Last update: **2021/10/16 21:50**

