



Display Animation in OnyxReport


To display an animation (e.g. created in module [V.T.O.3D](#) or [Aligner_3D](#)) in [OnyxReport](#), the animation needs to be saved in the module together with the corresponding finding. This has to be activated by a right mouse click on button [As Finding] before.

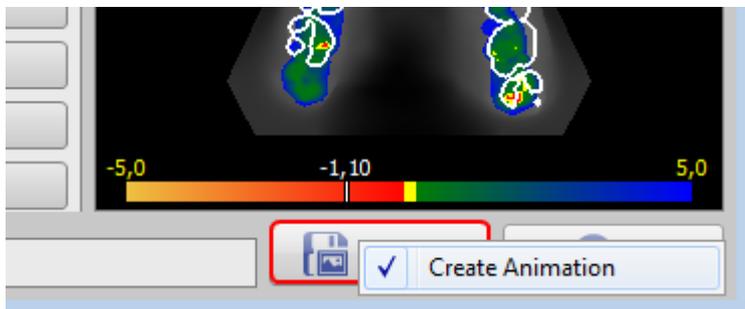

Saved animations are listed in the document tree on tab [Patient].

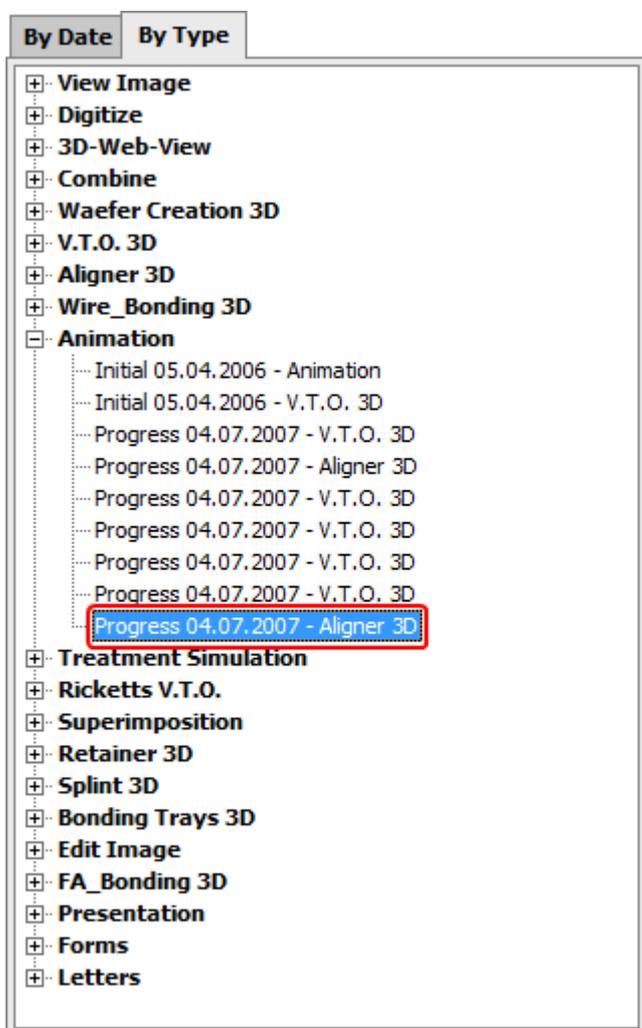

When adding the finding to [OnyxReport](#), checkbox [3D View] has to be activated before generation the report.

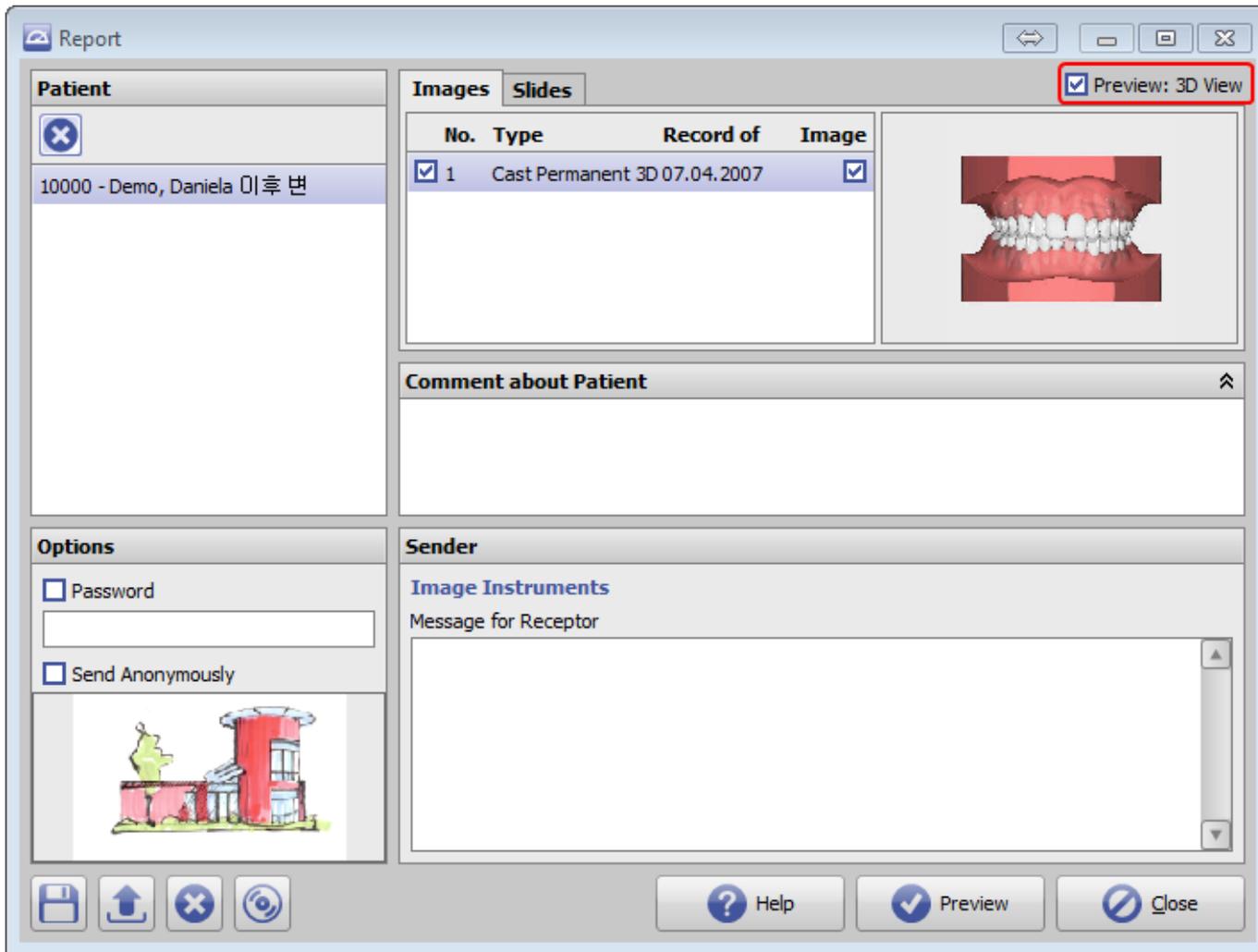

Thereafter, in the [OnyxReport](#) full screen view the animation can be played via contextmenu the same way as it is possible in the OnyxCeph³™ fullscreen view.

From:
<https://onyxwiki.net/> - **[OnyxCeph³™ Wiki]**

Permanent link:
https://onyxwiki.net/doku.php?id=en:workflow_0023

Last update: **2016/11/30 08:52**

