Hauptmenü-Eintrag Patient | Patient löschen

Soll ein Patienten-Datensatz vollständig aus der Datenbank entfernt werden, müssen zunächst sämtliche zum Patienten vorhandene Bild-Datensätze auf den Registerkarten Bilder (2D|3D) und Papierkorb (2D|3D) gelöscht werden. Erst wenn keine Bild-Datensätze zum aktiven Patienten-Datensatz mehr vorhanden sind, wird die Menüoption [Patient|Patient löschen] aktiv.

(1) Alle Bilder des Patienten auf Reiter |Bilder| löschen

🔤 OnyxCe	ph ^a ™ Lab - Ray, Irin - 70054						_		×
			Licen:	se 📃 Rel	ease (Updat	es are available	e) 🔲 Support	æ i	Language
	Practice Name	Patient				_			
	Image Instruments 🔹 🔹 👔	70054: Ray, Irin	• 20	Patient	Images	Letters	Presentation	Recy	cle Bin
8	😥 ×2 🎤 🏡 🎗 🏦 🎇 💷 🚺) v 🐻 v	2	D Data 3	D Data		:0 • 🖂		• ?

(2) Alle Präsentationen des Patienten auf Reiter |Präsentation| löschen

🖾 Onyx	Ceph ^a ™ Lab - Ray, Irin - 70054						_		×
				License	📕 Release (Upda	tes are available)) 🔲 Support	🕀 L	anguage
	Practice Name		Patient			_			
	Image Instruments	• 🕅	70054: Ray, Irin	▼ 🧸	Patient Image	Eetters P	resentation	Recyc	cle Bin
•	113 T T T T					Slides M	1edia 🔛		• 🕜

(3) Alle Daten des Patienten auf Reiter |Papierkorb| löschen

🖾 Ony	rxCeph³™ Lab - Ray, Irin - 70054					-		×
			u 🗆	License 🔲 Release (Upda	ates are available)	Support	🌐 La	anguage
	Practice Name	Patient		_				
	Image Instruments	 70054: Ray 	Irin 🔻 🗜	Q Patient Image	s Letters Pro	esentation	Recyc	le Bin
0	2 2		Glob	bal Data 2D Images	3D Images Sli	ides 🖻	۲	• ?

(4) Patient löschen

onyxCeph³™ Lab - Ray, Irin - 70054			1										_		×			
Practice Name								Licens	e 📕 Rel	ease (Upda	ates are availa	ble) 📕 S	upport	🇮 La	nguage			
				Patient														
	<u>P</u> atient	•	8 New Patient	1	Ctrl+N	• 🖄	70054: Ray, Irin			▼ 20	Patient	Image	s Letters	Presenta	ation		e Bin	
6	Edit	<u>S</u> elect Patier	Select Patient		•					Global D	ata 2DI	Images	3D Images	Slides			. 0	
	View			Delete Patient	<u>D</u> elete Patient									oo magaa	Distas			
	Communication	•		N <u>e</u> w Image		•												
	<u>G</u> o To	о То 🔹 🕨		Coloret Describes	ch	a colum												
	Options			Seject Practice	Sn	IIIT+CUI+P												
	Help	+																
	E <u>x</u> it																	

From: https://www.onyxwiki.net/ - [OnyxCeph^{3™} Wiki]

Permanent link: https://www.onyxwiki.net/doku.php?id=mm_patient_delete



Last update: 2023/04/05 11:19