# Systemoptionen - Tab |Patientendaten|

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The screenshot shows the 'System Options' dialog box with the 'Patient Data' tab selected. The left sidebar contains the following categories: Patient Data, Image Types, Sessions, Galleries, Printouts, Forms, Display, Interfaces, Third Party Tools, Event, Upload Data, and Email. The main content area is divided into three sections:

- Displayed Image Types:** Shows 'Left:' and 'Right:' image preview icons. Radio buttons are set to 'Latest Image' (selected) and 'Oldest Image'. A checkbox for 'Preview Images as Thumbnails (Fast Load but Low Quality)' is unchecked.
- VDDS-Media:** Includes a checked checkbox for 'Adoption of Master Data by VDDS-Media'. Below it, 'Adopt Master Data' is selected. A 'Delete old Entries' button is present. Under 'Image Information Provided for VDDS-Media', 'Slides with Tracing, Image Size 220 Pixels Side Length', '2D Original Images', and '3D Objects (Original)' are checked, while 'Projects (Show Information Only)' is unchecked. There are dropdown menus for 'Hand over Information to:', 'Hand over Documents to:' (set to 'my\_PVS'), and 'Hand over Analysis Results to:'. 'Practice Number' is set to '1' and 'Program Start' is set to 'normal'.
- Miscellaneous:** Features an 'Edit...' button for 'Markers', a text field for 'Consecutive Patient ID' containing '70057', and an unchecked checkbox for 'Capitalize First Letter of Name Fields'.

At the bottom right, there are 'Help' and 'Close' buttons.

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